

ONETOUCH®

UltraSmart

Meter and Electronic Logbook In One



**Owner's
Booklet**

Dear OneTouch® UltraSmart® System Owner:

You have chosen one of the best blood glucose monitoring systems available. This booklet has important information you must know about the OneTouch® UltraSmart® Blood Glucose Monitoring System. Read it carefully.

Blood glucose monitoring plays an important role in diabetes control. A long-term study showed that keeping blood glucose levels close to normal can reduce the risk of diabetes complications by up to 60%*. The results you get with the OneTouch® UltraSmart® System can help you and your healthcare professional monitor and adjust your treatment plan to gain better control of your diabetes.

Your OneTouch® UltraSmart® System requires a small drop of blood, at least 1µL (microliter), and five seconds to provide you with an accurate plasma-calibrated test result.

The OneTouch® UltraSmart® Meter also has a memory that allows it to function as an “electronic logbook.” Now you can store vital everyday information such as insulin and oral medication doses, food intake, amount of exercise, illnesses, and other pertinent data. You can use your OneTouch® UltraSmart® System simply to test or to serve as a complete storehouse of information regarding your diabetes treatment plan.

A warranty registration card is included with your system. If you complete it and mail it to us, we will send you a gift. If you prefer to fill out a warranty card online, please visit www.LifeScan.com.

*American Diabetes Association position statement on the Diabetes Control and Complications Trial (1993).

INTENDED USE: The OneTouch® UltraSmart® Blood Glucose Monitoring System is intended to be used for the quantitative measurement of glucose in fresh capillary whole blood. The OneTouch® UltraSmart® System is intended for use outside the body (for *in vitro* diagnostic use) by people with diabetes at home and healthcare professionals in a clinical setting as an aid to monitor the effectiveness of diabetes control. The OneTouch® UltraSmart® System should not be used for the diagnosis of diabetes or testing of newborns.

The OneTouch® UltraSmart® Blood Glucose Monitoring System provides the user with electronic logbook functions, which serve to store such data as insulin and oral medication doses, food intake, amount of exercise, health information such as illnesses, and other pertinent data. The meter includes a data port that enables the user to download logbook data to a personal computer.

CAUTION: Before using any product to test your blood glucose, read all instructions and practice the test. Do all quality control checks as directed and consult with a diabetes healthcare professional. These recommendations apply to all blood glucose monitoring systems and are supported by the American Association of Diabetes Educators, the U.S. Food and Drug Administration, and the Advanced Medical Technology Association.

WARNING: Keep the meter and testing supplies away from young children. Small items such as the battery door, battery, test strips, lancets, protective disks on the lancets, stereo jack cover and the control solution vial cap are choking hazards.

Important Information

- **Dehydration:** Severe dehydration and excessive water loss may cause false low results. If you believe you are suffering from dehydration, consult a healthcare professional immediately.
- **Low glucose results:** If your test result is lower than 70 mg/dL or is shown as “LOW GLUCOSE”, it may mean hypoglycemia (low blood glucose). **This may require immediate treatment according to your healthcare professional’s recommendations.** Although this result could be due to a test error, it is safer to treat first and then repeat the test.
- **High glucose results:** If your test result is greater than 180 mg/dL or is shown as “HIGH GLUCOSE”, it may mean hyperglycemia (high blood glucose). If you do not have symptoms, first repeat the test. Your healthcare professional can work with you to decide what actions, if any, you should take if you continue to get results higher than 180 mg/dL or if you have symptoms.
- **Repeated unexpected results:** If you continue to get unexpected results, check your system with control solution. See Checking the System with Control Solution on page 12. If you are experiencing symptoms that are not consistent with your blood glucose results and you have followed all instructions in this booklet, call your healthcare professional. Never ignore symptoms or make significant changes to your diabetes control program without speaking to your healthcare professional.
- **Hematocrit:** A hematocrit (percentage of your blood that is red blood cells) that is either very high (above 55%) or very low (below 30%) can cause false results.

Table of Contents

THE OneTouch® UltraSmart® SYSTEM	1
OneTouch® UltraSmart® Meter.....	2
Navigation Buttons	3
SmartButtons®	4
Display Symbols	6
Display Contrast.....	6
OneTouch® Ultra® Test Strips.....	7
BEFORE TESTING	9
Checking the Display	9
Setting the Meter	9
Coding the Meter	10
Checking the System with Control Solution	12
TESTING YOUR BLOOD	19
Preparing the OneTouch® UltraSoft®	
Adjustable Blood Sampler	20
Lancing and Sampling from Your Fingertip.....	23
Lancing and Sampling from an Alternate Site	27
Step-by-Step Test Procedure	31
Discarding Used Test Strips and Lancets.....	37
Special Messages	38
COMPARING METER AND LABORATORY RESULTS ...	40
CARING FOR YOUR SYSTEM.....	42
Maintenance	42
Cleaning the OneTouch® UltraSoft®	
Adjustable Blood Sampler	42

Batteries	43
DISPLAY MESSAGES.....	46
WARNINGS AND ERROR MESSAGES	51
PROBLEMS AND POSSIBLE CAUSES.....	56
METER SET-UP.....	58
Entering Basic Set-up Mode.....	58
Selecting Display Language	59
Setting Date and Time.....	60
CUSTOM METER SET-UP SEQUENCE	62
Set-up Menu.....	62
Settings	66
Diabetes Pill Set-up.....	68
Insulin Set-up	70
Insulin Pump Set-up.....	72
Advanced Feature Set-up	72
YOUR ELECTRONIC LOGBOOK	77
Comments.....	77
Rules for Adding Comments	78
How to Add a Comment	78
Logbook Entries	82
Using the SmartButtons®	82
Rules for Logbook Entries.....	84
Entering the Date and Time	85
Exercise Logbook Entries	86
Health Logbook Entries.....	87

Table of Contents (continued)

Ketones.....	89
HbA1c	91
Microalbumin.....	91
Cholesterol	92
Blood Pressure	92
Eye Exam	93
Foot Exam	93
Weight/Height	94
Doctor Visit	94
Medication Logbook Entries.....	95
Food Logbook Entries.....	101
Adding, Changing, or Deleting	
Logbook Information	103
OneTouch® UltraSmart® FastFacts® FEATURE.....	108
FastFacts® Menu.....	109
Logbook Memory	110
Glucose by Meals	110
Glucose Analysis	111
Interactive Graph of All Results	111
Graph by Time of Day	112
Average of All Results.....	113
Average by Time of Day	114
Average by Exercise.....	114
Glucose Range Info.....	115
Insulin Intake.....	116

Hypoglycemia Information	120
Food Averages.....	121
Health Checks	121
Data Downloading.....	122
SPECIFICATIONS	125
PRODUCT SAFETY INFORMATION	126
EXPLANATION OF SYMBOLS ON THE METER.....	127
GUARANTEE	127
INDEX.....	128



THE OneTouch® UltraSmart® SYSTEM

The OneTouch® UltraSmart® System uses the latest blood glucose monitoring technology. It measures the glucose content of a blood sample by means of an electrical current produced in the test strip and is measured by the meter. Test results are “plasma-calibrated.” This makes it easier for you and your diabetes-care team to compare your meter results with laboratory tests. If you have been using another type of meter—one that provides whole-blood results—you may notice that your test results with the OneTouch® UltraSmart® System are about 12% higher.

The OneTouch® UltraSmart® Blood Glucose Monitoring System consists of three main products: the OneTouch® UltraSmart® Meter, OneTouch® Ultra® Test Strips, and OneTouch® Ultra® Control Solution. These products have been designed, tested, and proven to work together as a system to produce accurate blood glucose test results. **Use no other test strips or control solution other than the OneTouch® Ultra® brand with your meter.**

Your system kit includes:

- OneTouch® UltraSmart® Meter
- OneTouch® Ultra® Control Solution
- OneTouch® UltraSoft® Adjustable Blood Sampler
- OneTouch® UltraClear® Cap
- OneTouch® UltraSoft® Sterile Lancets
- Owner's Booklet
- Carrying Case
- Quick Reference Guide
- Quick Start Guide
- Warranty Registration Card
- Two AAA Alkaline Batteries (Installed)

OneTouch® Ultra® Test Strips sold separately

OneTouch® UltraSmart® Meter

OneTouch® Ultra® Test Strip

Test Port

The meter turns on when you insert the OneTouch® Ultra® Test Strip here.

Data Port

Used to download logbook memory to a computer.

Display with Backlight

Test results, messages, symbols, and other data appear here.

SmartButtons®

(See pages 4–5 for description and use.)



Navigation Buttons

Note: Press and release buttons. **Do Not** hold buttons down for an extended period of time.

Back Button (Off Button)

*Go back to previous display on certain screens.
Press and hold for two seconds to turn off the meter.*

Arrow Button

Scroll through menus, code numbers, and logbook.

OK Button (Backlight Button)

*Accept entries and move to next display.
Press and hold for two seconds to turn on backlight.*

SmartButtons®

The SmartButtons® are your keys to getting the most out of your OneTouch® UltraSmart® Meter. They allow you to enter information about your diabetes management plan into the large logbook memory. There are SmartButtons® for exercise, health, medication, and food. The FastFacts® SmartButton® is used to view the records in the logbook memory. You may press any SmartButton® to turn on the meter and move directly to the category.

Exercise SmartButton®

Enter exercise amount and level.

Food SmartButton®

Enter meal and snack information.



Health SmartButton®

Enter information on health.

Medication SmartButton®

Enter insulin and diabetes pill information.




FastFacts® SmartButton®

Review test results and logbook memory.



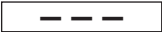

Note: For complete information on using the Exercise, Health, Medication, and Food SmartButtons®, please see pages 82–107. See pages 108–122 for more information on the FastFacts® Feature and the FastFacts® SmartButton®.

Display Symbols

These symbols guide you while using your meter:

Symbol	Meaning
▲	Scroll (move) up only.
▼	Scroll down only.
▲▼	Scroll up or down.
▲▼	Scroll up or down.
	No entry.
	Shows where you are on the display. Press the OK button  to accept highlighted area.

Display Contrast

You can change the contrast of the meter display. To make it darker, press and hold both the Back button  and the “up” Arrow button . To make it lighter, press and hold both the Back button  and the “down” Arrow button .

OneTouch® Ultra® Test Strips

The OneTouch® UltraSmart® System measures the amount of glucose in whole blood. Blood is applied to the **Top Edge** of the OneTouch® Ultra® Test Strip and is automatically drawn into the reaction cell where the reaction takes place.

Top Edge

Apply a drop of blood to the narrow channel here in the top edge of the test strip.

Confirmation Window

Check here to confirm if enough blood has been applied.

Contact Bars

Insert this end of the test strip, contact bars facing up, into the meter. Push it all the way in until it will go no further.



WARNING: Do Not swallow test strips. The test strip vial may contain drying agents that are harmful if swallowed and may cause skin or eye irritation.

See pages 19–39, *Testing Your Blood*, for complete instructions.

Important Test Strip Information

- Store test strip vials in a cool, dry place below 86°F (30°C). Keep away from direct sunlight and heat. **Do Not** refrigerate.
- Store test strips in their **original vial only**. To avoid damage or contamination, **Do Not** transfer test strips to any other place.
- After removing a test strip from the vial, replace the vial cap immediately and close it tightly.
- Use each test strip immediately after removing it from the vial.
- **Do Not** use test strips from any vial that is damaged or left open to air.
- Write the discard date (6 months after first opening the vial) on the vial label when you first open it. Discard remaining OneTouch® Ultra® Test Strips after the discard date.
- **Do Not** use test strips beyond the expiration (printed on the package) or discard date, whichever comes first, because they may cause inaccurate results.
- Avoid getting dirt, food or liquids on the test strip. With clean, dry hands you may touch the test strip anywhere on its surface.
- **Do Not** bend, cut, or alter OneTouch® Ultra® Test Strips in any way.
- OneTouch® Ultra® Test Strips are for single use only. **Never reuse a test strip that has had either blood or control solution applied to it.**
- Apply only OneTouch® Ultra® Control Solution or a blood sample to the test strip.
- Refer to additional information in the OneTouch® Ultra® Test Strip package.

BEFORE TESTING

Checking the Meter Display

Each time you turn the meter on, the display background will quickly change from dark to light and form the words “OneTouch UltraSmart”. This tells you that the system is performing several self-checks to confirm that the meter is working properly. If, at any time during the life of the meter, these graphics appear to be different, call **LifeScan Customer Service at 1 800 227-8862**.

ONE TOUCH
UltraSmart

Setting the Meter

The OneTouch® UltraSmart® Meter is set at the factory to display results in the English language using milligrams per deciliter (mg/dL) as the unit of measure. The time and date are also factory-set in Pacific Standard Time. If you are using the meter for the first time and simply wish to test, insert a test strip into the test port and begin by coding the meter. If you wish to change the date, time, or language, see *Meter Set-up*, pages 58–61. To customize your meter, see *Custom Meter Set-up Sequence*, pages 62–76.

Note: It is important that the time of day is correct and that your medication set-up and mealtimes are set according to your personal daily schedule. Otherwise, some FastFacts® information will not be correct.

Coding the Meter

Code numbers are used to calibrate the OneTouch® UltraSmart® Meter with the OneTouch® Ultra® Test Strips for accurate results. You must code the meter before using it for the first time and then every time you change to another vial of test strips.


CAUTION: Matching the code on the meter and the code on the test strip vial is essential to obtain accurate results. Each time you test, check to make sure the code numbers match.



STEP 1 Enter the Code Mode.

Insert a OneTouch® Ultra® Test Strip to turn on the meter. Push it all the way in until it will go no further. “OneTouch UltraSmart” will appear on the display briefly. The first time you turn the meter on, three dashes [---] will appear, showing that there is no code stored in the memory. Follow Step 4 to code the meter.

STEP 2 See the Code Number.

The code number will appear on the display. (After every 25 tests the meter will display the code number and will prompt “Check Code Press OK”. Press the OK button  if the code number is still correct.)




(Example)

STEP 3 Match the Code Numbers.

Compare the code number on the meter display with the code number on the test strip vial. If the two code numbers match, you may begin testing. If they do not match, follow Step 4.




(Example)

Note: The code number appears on the display briefly. If you were unable to change the code in time or entered a code number in error, press the Back button  to return to Step 3.



(Example)

STEP 4 Code the Meter.

Move up or down using the Arrow button  to the code number that matches the code number on the test strip vial. The new code number will flash on the display for about three seconds, after which the meter will prompt you to “Apply Blood”.

Checking the System with Control Solution

Why you should do a control solution test:

- OneTouch® Ultra® Control Solution is used to check that the meter and the test strips are working together as a system and that you are performing the test correctly.
- It is very important that you do this simple check routinely to make sure you get an accurate result.



Before you begin:

- Use only OneTouch® Ultra® Control Solution.
- Check the expiration date on the control solution vial. Record the discard date (date opened plus six months) on the vial label. **Do Not** use after expiration or discard date, whichever comes first.
- Control solution, meter, and test strips should be at room temperature (between 68° and 77°F/between 20° and 25°C) before testing with control solution (see *Specifications* section on page 125 for blood testing temperature ranges).
- Shake the vial, discard the first drop of control solution, and wipe off the dispenser tip to ensure a good sample and an accurate result.
- Store the control solution tightly closed at temperatures below 86°F (30°C). **Do Not** refrigerate.

WARNING:

- **Do Not swallow control solution; it is not for human consumption.**
- **Do Not apply control solution to the skin or eyes as it may cause irritation.**

When you should do a control solution test:

- To practice the test process instead of using blood.
- When you open a new vial of test strips.
- Once a week.
- Whenever you suspect that the meter or test strips are not working properly.
- If you have had repeated unexpected blood glucose results (as described on page v).
- After dropping the meter.

Note: Refer to additional information in the OneTouch® Ultra® Control Solution package.

STEP 1 Insert a Test Strip into the Meter.

To turn on the meter, insert a test strip, contact bars end first and facing up, into the test port. Push it all the way in until it will go no further.


“OneTouch UltraSmart” will appear briefly on the display.


Then “Code 9 (Example), Check Code” is displayed. Be sure the meter and test strip codes match. If they do not, code the meter correctly (see pages 10–12).




(Example)

STEP 2 Select “Control Solution”.

“Apply Blood” will appear on the display first. Use the Arrow button  to select “Control Solution”. The meter will automatically identify the test as a control solution test in the logbook.

There are three choices in test mode. Use the Arrow button  to select one. **Do Not** apply a sample until your choice appears on the display.


- “Apply Blood” for fingertip blood sample.
- “Alternate Site” for forearm and palm blood samples.
- “Control Solution” for checking the system.

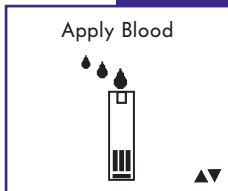

Note: You must select “Control Solution” to distinguish control solution tests from blood glucose tests. Marked control solution tests will not be included in your averages. **Do Not** use Control Solution mode for testing blood.

STEP 3 Apply Control Solution to the Test Strip.


To ensure an accurate control solution test, shake the vial, discard the first drop and wipe the dispenser tip before you begin.

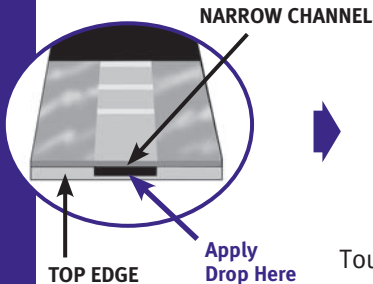

Touch and hold a drop of control solution where the narrow channel meets the **TOP EDGE** of the strip.

Note: If you do not apply the control solution within three minutes, the meter will turn itself off. Remove the test strip and insert it back into the test port to restart the test procedure.

CONFIRMATION WINDOW

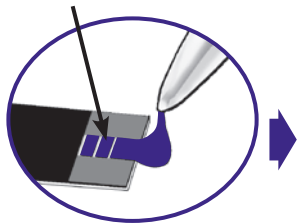

The control solution will be drawn into the narrow channel. When the confirmation window is full, the meter will count down from 5 to 1.



(Example)

WARNING:

If mg/dL does not appear with the Control Solution result, call LifeScan Customer Service at 1 800 227-8862.

The control solution test result will appear on the display.

STEP 4

Compare Control Solution Result to the Control Range Printed on the TEST STRIP VIAL.

CAUTION: The control solution range printed on the test strip vial is for OneTouch® Ultra® Control Solution only. It is not a recommended range for your blood glucose level.

CONTROL RANGE



(Example)

If the results are not within the control range printed on the test strip vial, the meter and strips may not be working properly. Repeat the control solution test. When using Control Solution mode, very high or very low readings will be displayed as a number rather than a “HIGH” or “LOW” warning screen.

Control solution out-of-range results may be due to:

- Incorrect unit of measure.
- Error in performing the test.
- Failure to shake the control solution vial well.
- Improper meter coding.
- Test strip deterioration.
- Meter malfunction.
- Failure to discard the first drop of control solution and wipe the dispenser tip clean.
- Expired or contaminated control solution.
- The meter, test strips, or control solution are too warm or too cold.
- Testing blood in Control Solution mode.

CAUTION: If you continue to get control solution test results that fall outside of the range printed on the test strip vial, the system may not be working properly. **Do Not** use the meter. Call **LifeScan Customer Service at 1 800 227-8862.**

TESTING YOUR BLOOD

Read this section and the test strip package insert carefully before testing. Make sure you have all items needed to test.


- OneTouch® UltraSmart® Meter
- OneTouch® Ultra® Test Strips
- OneTouch® UltraSoft® Adjustable Blood Sampler
- OneTouch® UltraClear® Cap (for forearm and palm sampling)
- OneTouch® UltraSoft® Sterile Lancets

CAUTION: It is important to learn the correct blood application technique (see pages 33–35). If you do not apply blood to the test strip correctly you may get an inaccurately low or high test result or an error message. LifeScan recommends that you develop good technique by practicing with control solution prior to using the OneTouch® UltraSmart® Meter for the first time.

CAUTION: If you cannot test due to a problem with your testing supplies, contact your healthcare professional or **LifeScan Customer Service at 1 800 227-8862**. Failure to test could delay treatment decisions and lead to a serious medical condition.

Preparing the OneTouch® UltraSoft® Adjustable Blood Sampler (Sampler)

OneTouch® UltraSoft® Adjustable Blood Sampler

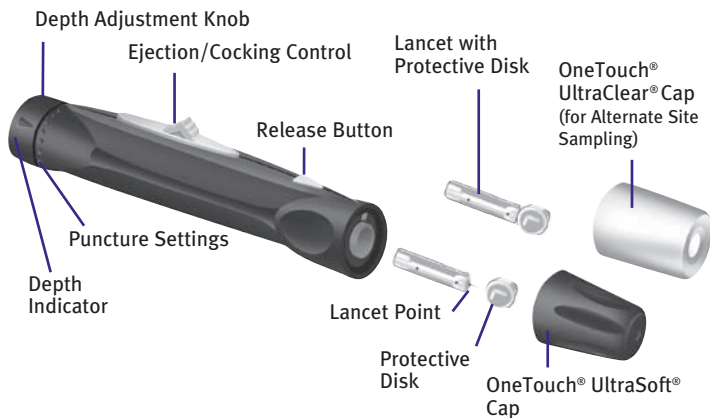

CAUTION: To reduce the chance of infection:

- Never share a lancet or the **Sampler** with anyone.
- Always use a new, sterile lancet. Lancets are for single use only.
- Keep the OneTouch® UltraSmart® Meter and **Sampler** clean.
- Make sure to wash the puncture site with soap and water before testing.

Note: If you **Do Not** have a OneTouch® UltraSoft® Adjustable Blood Sampler, please refer to the instructions that came with your lancing device.

Choosing the right sampling site at the right time:

The OneTouch® UltraSmart® Meter allows you to sample blood from your fingertip, forearm or palm. At times, results obtained at the forearm or palm may be different from a fingertip measurement. Talk to your healthcare professional before you begin using your forearm or palm for sampling.

If you are testing:	Use blood sample from your:
<p>Routinely before meals</p> <p>Prior to or more than two hours after:</p> <ul style="list-style-type: none">• a meal• a rapid-acting insulin injection or insulin pump bolus• exercise	<p>Fingertip, forearm, or palm</p>
<p>When your blood glucose is changing rapidly, such as:</p> <ul style="list-style-type: none">• within two hours after a meal• within two hours after a rapid-acting insulin injection or insulin pump bolus, or• during or within two hours after exercise <p>When you are concerned about the possibility of hypoglycemia (low blood sugar)</p>	<p>Fingertip</p>

CAUTION: Do Not test on your forearm or palm when:

- You think your blood glucose is rapidly falling, such as within two hours of exercise or a rapid-acting insulin injection or insulin pump bolus. Testing with a fingertip sample may identify hypoglycemia or an insulin reaction sooner than testing with a forearm or palm sample.
- It has been less than two hours after a meal, a rapid-acting insulin injection or insulin pump bolus, physical exercise, or you think your glucose level is changing rapidly.
- You are concerned about the possibility of hypoglycemia or an insulin reaction, such as when driving a car. This is especially important if you suffer from hypoglycemia unawareness (lack of symptoms to indicate an insulin reaction).

Remember: Consult with your healthcare professional before using your forearm or palm for testing.

Choose a different puncture site each time you test. Repeated punctures in the same spot may cause soreness and calluses.

If bruising occurs at an alternate site or you have difficulty getting a sample, consider sampling from a fingertip instead. You may want to review the choice of sites with your healthcare professional.


Preparing Your Sample Site

Before you test your blood glucose, wash your hands and forearm (if applicable) thoroughly with warm, soapy water. Rinse and dry thoroughly.



Lancing and Sampling from Your Fingertip


STEP 1 Remove the OneTouch® UltraSoft® Cap by Twisting it Off.

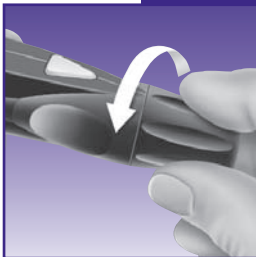




STEP 2 Insert a Sterile Lancet Into the Sampler.

Insert the lancet into the holder and push in firmly. Twist the protective disk until it separates from the lancet and save the disk for later use. **Do Not** twist the lancet.

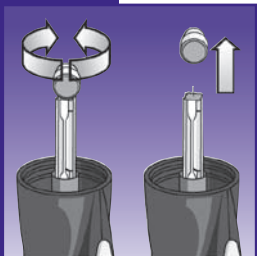

STEP 3 Replace the OneTouch® UltraSoft® Cap by Twisting it Back On Until it is Snug.

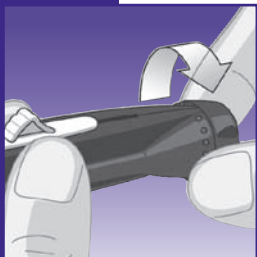

STEP 4 Adjust the Depth Setting.

If necessary, twist the depth adjustment knob toward the smaller dots on the **Sampler** for a shallow puncture or toward the larger dots for a deeper puncture.

STEP 5 Cock the Sampler.

Slide the ejection/cocking control back until it clicks. If it does not click, it may have been cocked when you inserted the lancet.



STEP 6 Puncture Your Finger.

Hold the **Sampler** firmly against the side of your finger. Press the release button. Remove the **Sampler** from your finger.



Note: Do Not use the OneTouch® UltraClear® Cap on your finger.

STEP 7**Get a Round Drop of Blood.**

Gently squeeze and/or massage your fingertip until a round drop of blood **at least 1 μ L (● sample size)** forms on your fingertip. **Do Not** squeeze excessively on the puncture site.




If the blood smears or runs, **Do Not** use that sample. Dry the area and gently squeeze another drop of blood or puncture a new site.

Lancing and Sampling From an Alternate Site

Sampling from your forearm or palm allows you to use your fingertips less often. You may find that obtaining a blood sample from an alternate site is less painful than using a fingertip. Getting a blood sample from your forearm or palm is different than getting a sample from your fingertips.

Forearm sampling

Choose a fleshy area of the forearm away from bone, visible veins and hair. Sometimes there is less blood flow to the forearm than to the fingertips. To help you get a large enough drop of blood, you may gently massage or apply warmth to the site to increase blood flow.

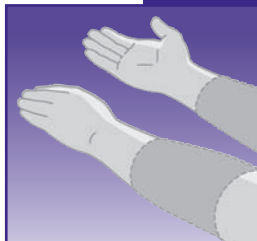


Forearm

Palm sampling

Choose a fleshy area on the palm below your thumb or pinky finger. Select a spot with no visible veins and away from deep lines which may cause your blood sample to smear.



Palm

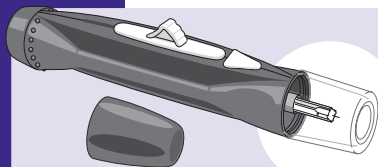

Blue cap for fingertip sampling only

OneTouch® UltraClear® Cap is used for forearm and palm sampling only. Replace the blue OneTouch® UltraSoft® Cap with the OneTouch® UltraClear® Cap.



STEP 1

Remove the OneTouch® UltraSoft® Cap by Twisting it Off.




STEP 2

Insert a Sterile Lancet Into the Sampler.

Insert the lancet into the holder and push in firmly. Twist the protective disk until it separates from the lancet and save the disk for later use. **Do Not** twist the lancet.


STEP 3

Twist On the OneTouch® UltraClear® Cap Until it is Snug.

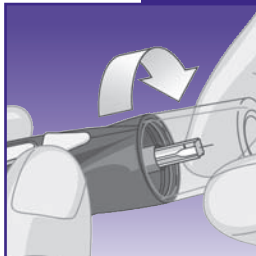**STEP 4**


Adjust the Depth Setting.

You may have to adjust the **Sampler** to a deeper setting to get a large enough drop of blood from your forearm or palm. Twist the depth adjustment knob toward the larger dots to increase the depth.

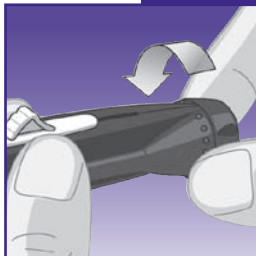**STEP 5**

Cock the Sampler.

Slide the ejection/cocking control back until it clicks. If it does not click, it may have been cocked when you inserted the lancet.

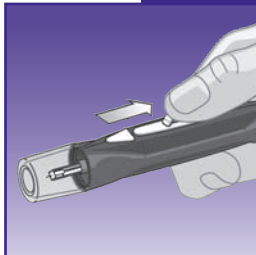



STEP 6

Puncture Your Forearm or Palm.

Press and hold the **Sampler** against your forearm or palm for a few seconds, then press the release button.

Keep holding the **Sampler** and OneTouch® UltraClear® Cap against your skin until a round drop of blood forms under the cap. Maintain pressure until the drop of blood is **at least 1 µL (●sample size)**.



STEP 7

Remove the Sampler.

Carefully lift the sampler away from your skin. **Do Not** smear the blood sample.


Note:

- You may need to wait a little longer to get a large enough drop of blood from the forearm or palm. **Do Not** squeeze the site excessively.
- If the sample drop of blood runs or spreads due to contact with hair or with a line in your palm, **Do Not** use that sample. Try puncturing again in a smoother area.
- Remember: You may have to adjust the **Sampler** to a deeper setting to get a large enough drop of blood (●sample size).


Step-by-Step Test Procedure

STEP 1 Insert Test Strip.

Insert a test strip, contact bars end first and facing up, into the test port. Push it all the way in until it will go no further. The meter will turn on automatically. “OneTouch UltraSmart” will appear briefly on the display. Then the code number will appear. Be sure the meter and test strip codes match. If they do not, code the meter correctly (see pages 10-12 for *Coding the Meter*).

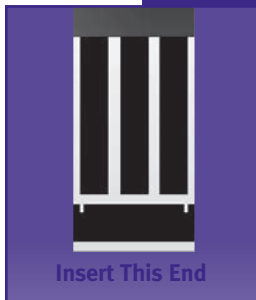

STEP 2 Select Test Mode.

Note: There are three choices in test mode. For blood testing, use the Arrow button  to select:

- “Apply Blood” for fingertip blood sample
- “Alternate Site” for forearm and palm blood samples

Do Not use Control Solution mode when testing blood.

- “Control Solution” is for checking the system


Apply Blood



“Apply Blood” will appear on the display first. If you are testing with a fingertip blood sample, proceed to Step 3.

Alternate Site



If you are testing with a blood sample from the forearm or palm, use the Arrow button  to scroll to “Alternate Site”.

Before you test with blood, make sure “Apply Blood” or “Alternate Site” is displayed on the screen. Correct test mode selection is important to ensure all blood and control solution tests are properly included or excluded from averages. Failure to select the correct test mode may cause blood averages to be incorrect.

STEP 3 Prepare to Apply Sample.

The drop of blood should be next to the top edge of the test strip. **Blood should not be applied to the flat surface of the test strip.**

When applying a drop of blood from your forearm or palm, keep your forearm or palm steady and bring the top edge of the test strip to the drop of blood with your other hand.



Fingertip



Forearm




Palm



STEP 4 Apply Sample.

Obtain a round drop of blood using the **Sampler**. The blood sample must be at least **1 μ L** in volume (● sample size) to fill the confirmation window.

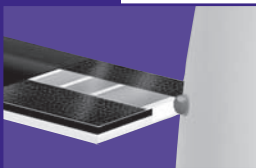

Touch and hold the drop of blood to the narrow channel in the top edge of the test strip.




- **Do Not** apply sample to the front or back of the test strip.
- **Do Not** push your finger against the test strip.
- **Do Not** apply a smeared sample.



Hold the blood drop to the top edge of the test strip **until the confirmation window is full before the meter begins to count down.**

If the confirmation window does not fill completely before the meter begins to count down, **Do Not** add more blood to the test strip; discard the test strip and retest. If you have trouble filling the test strip, call **LifeScan Customer Service at 1 800-227-8862** for assistance.

If the confirmation window is not full, you may get an error message or an inaccurate test result.

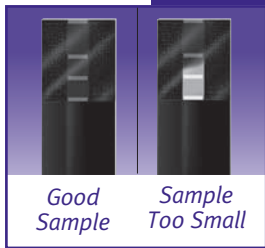


Note: If you do not apply a blood sample within three minutes, the meter will turn off automatically. Remove the test strip and then insert it back into the meter to turn the meter back on.

STEP 5 Accurate Results in Just 5 Seconds.

Your blood glucose test result will appear after the meter counts down from 5 to 1.

The test results are automatically stored in the electronic logbook (memory). You may now:

- Use the Arrow button **8** to compare results (see “Additional Feature” which follows).
- Add comments to your result.
- Remove the test strip to turn off the meter.

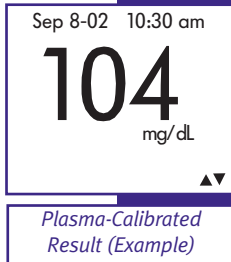

WARNING: If mg/dL does not appear with the test result, call LifeScan Customer Service at 1 800-227-8862.

CAUTION: If you test at the low end of the operating range (43°F/6°C) and your glucose is high (over 180 mg/dL), the reading on your meter may be lower than your actual glucose. In this situation, repeat the test in a warmer environment with a new test strip as soon as possible.

Note: If “Comment? Press OK” appears at the bottom of the display, your test result is outside the range of 90–130 mg/dL or a personal glucose range you set (see pages 62–76, *Custom Meter Set-up Sequence*). To add a comment to the test result, see pages 77–81, *Your Electronic Logbook*.

Your Result: **104**

Previous
Sep 13-02
9:55 pm


114

Bef Brkft
Avg.
14 days

95



Additional Feature

After you complete a blood glucose test, you can compare the result with your last test and your average for this time of day by pressing the “down” Arrow button .


(Example)

Discarding Used Test Strips and Lancets

CAUTION: It is important to discard the used lancet carefully after each use to avoid unintended lancet stick injuries. Used test strips and lancets may be considered biohazardous waste in your area. Be sure to follow your local regulations for proper disposal.


STEP 1 Remove the OneTouch® UltraSoft® Cap from the Sampler.

Twist the OneTouch® UltraSoft® Cap counterclockwise. (Optional: Replace the protective disk on the used lancet by placing it on a hard surface and pushing the lancet tip into the disk.)

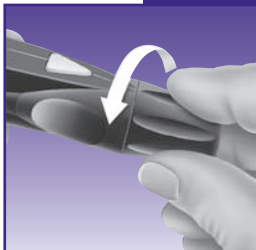


STEP 2 Eject the Lancet.


Point the **Sampler** down and away. Push the release button to ensure that the **Sampler** is not cocked. Push forward on the ejection/cocking control and eject the lancet directly into a container for sharp objects. Used test strips can go into the same container. Return the ejection/cocking control to the middle position. Replace the OneTouch® UltraSoft® Cap.

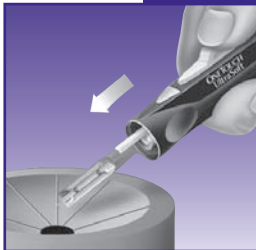

Special Messages

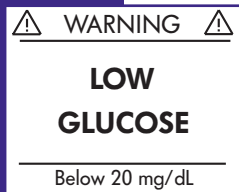

The OneTouch® UltraSmart® Meter displays results between 20 mg/dL and 600 mg/dL. If your test result is lower than 20 mg/dL, “LOW GLUCOSE” will appear on the meter display. **This may mean severe hypoglycemia (low blood glucose) that requires immediate treatment according to your healthcare professional’s recommendations.** Although this message could be due to a test error, it is safer to treat first and then repeat the test.

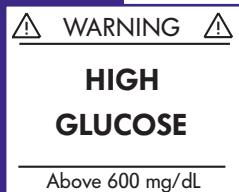

If your blood glucose test result is above 600 mg/dL, “HIGH GLUCOSE” will appear on the meter display. **This may mean severe hyperglycemia (high blood glucose). You should recheck your blood glucose level. If “HIGH GLUCOSE” again appears on the display, call your healthcare professional immediately.**


When your blood glucose test result is outside the range of 90–130 mg/dL or the personal glucose range you set in Custom Meter Set-up Sequence, the meter will prompt you to add a comment.



When your blood glucose test result is in the range of 240 mg/dL to 600 mg/dL, “Check Ketones” will appear on the meter display. **This message does not mean that the system detected ketones but that testing with a ketone test strip may be advisable. Consult your healthcare professional about when and how to test for ketones.** “Comment? Press OK” will also appear.



When your blood glucose result is from 20 mg/dL through 59 mg/dL, “Do you need a snack?” will appear on the display with the test result, reminding you that a meal or snack may be necessary.

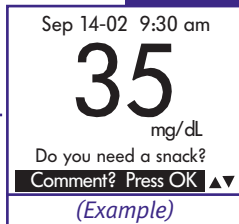

COMPARING METER AND LABORATORY RESULTS

OneTouch® UltraSmart® Meter test results and laboratory test results are both expressed in plasma-equivalent units. However, your meter result may differ somewhat from your laboratory result due to normal variation. Meter results can be affected by factors and conditions that do not affect laboratory results in the same way. Your OneTouch® UltraSmart® Meter glucose value should agree with a laboratory measurement within $\pm 20\%$ most of the time under normal conditions. A result within that range is considered accurate when testing with the OneTouch® UltraSmart® Meter. However, results can differ by more than $\pm 20\%$ in some situations. See the OneTouch® Ultra® Test Strip package insert for typical accuracy and precision data and for important information on “Limitations of Procedure.”

To make an accurate comparison between meter and laboratory results, follow these guidelines:

Before you go to the lab:

- Perform a control solution test to make sure the meter is working properly.
- It is best to fast for at least eight hours before doing comparison tests.
- Take your meter with you to the lab.

While at the lab:

- Make sure that the meter test and the lab test are performed within 15 minutes of each other.
- Wash your hands before obtaining a blood sample.
- Use only fresh capillary blood.

You may still have a variation between the results because blood glucose levels can change significantly over short periods, especially if you have recently eaten, exercised, taken medication, or experienced stress.¹ In addition, if you have eaten recently, the blood glucose level from a fingerstick can be up to 70 mg/dL higher than blood drawn from a vein (venous sample) used for a lab test.² Therefore, it is best to fast for eight hours before doing comparison tests. Factors such as the amount of red blood cells in the blood (a high or low hematocrit) or the loss of body fluid (severe dehydration) may also cause a meter result to be different from a laboratory result.

References

1. Surwit, R.S., and Feinglos, M.N.: *Diabetes Forecast* (1988), April, 49–51.
2. Sacks, D.B.: “Carbohydrates.” Burtis, C.A., and Ashwood, E.R. (ed.), *Tietz Textbook of Clinical Chemistry*. Philadelphia: W.B. Saunders Company (1994), 959.

CARING FOR YOUR SYSTEM

Maintenance

Your OneTouch® UltraSmart® Meter does not require special maintenance or cleaning. Take care to avoid getting dirt, dust, blood, control solution, or liquids inside the meter through the test port or data port. LifeScan recommends that you store the meter in its carrying case after each use.

A cloth dampened with water and mild detergent can be used to wipe down the outside of the meter. Your OneTouch® UltraSmart® Meter is a precision instrument. Please handle it with care.

Do Not use alcohol or any other solvent to clean your meter.

Cleaning the OneTouch® UltraSoft® Adjustable Blood Sampler


Clean the **Sampler** and OneTouch® UltraSoft® and OneTouch® UltraClear® Caps with soap and water. To disinfect the **Sampler**, prepare a disinfectant solution of one part household bleach to 10 parts water. Dampen a cloth with this solution and wipe the **Sampler** thoroughly. **Soak only the cap or caps** for at least 30 minutes in the disinfectant solution. **Do Not** soak the **Sampler** in liquid. Rinse the **Sampler** and cap with water and dry thoroughly.

Batteries


Your OneTouch® UltraSmart® Meter comes with two AAA alkaline batteries already installed. The batteries will provide you with enough power to perform up to 540 tests, depending on backlight use. When replacing the batteries, use only two AAA alkaline batteries.

The meter will alert you when the power is getting low by displaying two different messages:

1. When this message appears on the display, the batteries are low and should be replaced as soon as possible. The backlight will no longer turn on.

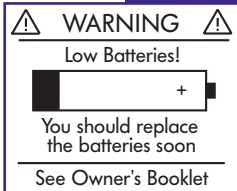

2. When this message appears on the display, you cannot test, enter data in your electronic logbook, or use the FastFacts® feature until you change the batteries.

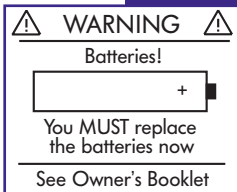

To replace the batteries, first turn off the meter. Turn the meter over and locate the battery compartment.



1. Open the battery compartment by pressing the tab to the left and lifting the battery compartment cover to remove it.



2. Remove the old batteries by pulling the black ribbon upward. Insert two AAA alkaline batteries on top of the black ribbon. Plus (+) and minus (-) signs will guide you in placing the batteries.



3. Insert the tabs of the battery cover into the slots on the left hand side of the battery compartment. Push down on the battery cover until you hear the tab click into place.

Note:

- Replacing the batteries does not affect the electronic logbook. However, the date and time settings may need to be updated.
- The first time you turn on the meter after replacing the batteries, the meter will go into the Set-up mode (see pages 58–61). At this time, you may update the date and time settings.

DISPLAY MESSAGES

Following is a summary of screen messages and symbols.

MESSAGE


WHAT IT MEANS

ACTION



Initial display that appears whenever the meter is turned on. It indicates that the system has performed several self-checks to confirm that the meter is working properly.


If, at any time during the life of the meter, these graphics appear to be different, call **LifeScan Customer Service at 1 800 227-8862.**

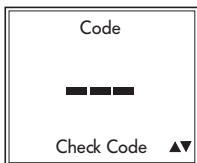

New meter not coded or meter lost power briefly and entered Set-up mode.

See *Coding the Meter*, pages 10–12.

MESSAGE

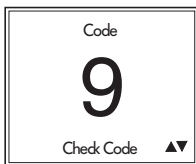

(Example)

WHAT IT MEANS

This is the code number stored in the meter.

ACTION



Make sure that this code number matches the code number on the vial of the test strips you are using.

CAUTION: Matching the code on the meter and the code on the test strip vial is essential to obtain accurate results. Each time you test, check to make sure the code numbers match.



(Example)


The meter is prompting you to confirm that the code number on the meter display matches the code number on the vial of test strips you are currently using.

Check to be sure the code number on the display matches the code number on the test strip vial. If they match, press the OK button  to confirm the code number. If they do not match, scroll to the correct number using the Arrow button .

MESSAGE


WHAT IT MEANS

ACTION

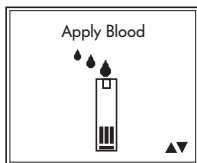

The system is ready to accept a blood sample from the fingertip.

You may now apply the fingertip blood sample.

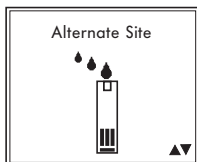

The system is ready to accept a blood sample from the forearm or palm.


You may now apply the forearm or palm blood sample.



The system is ready to accept a control solution sample.

You may now apply the control solution.

MESSAGE

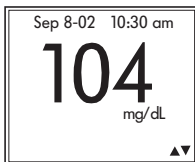

(Example)

WHAT IT MEANS

Five-second countdown. After the countdown, the meter will display the test result.

A test result in mg/dL.

ACTION

None required.


None required.

WARNING: If mg/dL does not appear with the test result, call LifeScan Customer Service at 1 800 227-8862.



(Example)

A test result outside of your target range with a message prompting you to add a comment.

If you wish to add a comment, press the OK button  (see page 78).

MESSAGE



(Example)



(Example)

WHAT IT MEANS

A blood glucose test result in the range of 240 mg/dL to 600 mg/dL with a suggestion to check your ketone levels.

When your blood glucose result is from 20 mg/dL through 59 mg/dL, "Do you need a snack?" will appear on the display with the test result, reminding you that a meal or snack may be necessary.

ACTION



Follow the instructions of your healthcare professional regarding ketone testing.

Follow the instructions of your healthcare professional.

WARNINGS AND ERROR MESSAGES

These messages help to identify certain problems but do not appear in all cases when a problem has occurred. Improper use may cause an inaccurate result without producing a warning or an error message. In the event of a problem, refer to the information under ACTION. Assistance is available from **LifeScan Customer Service at 1 800 227-8862. Contact us 7 days a week, 8 a.m. – 10 p.m. Eastern Time.**

MESSAGE

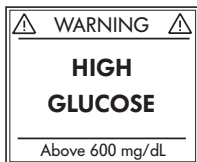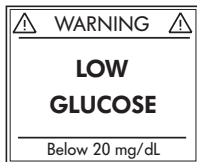

WHAT IT MEANS

You may have a very low blood glucose level, lower than 20 mg/dL.



You may have a very high blood glucose level, exceeding 600 mg/dL.

ACTION

Treat this condition immediately according to your healthcare professional's recommendations. Although this message could be due to a test error, it is safer to treat first and then repeat the test.

You should retest your blood glucose level and if "HIGH GLUCOSE" appears again, call your healthcare professional immediately.

MESSAGE

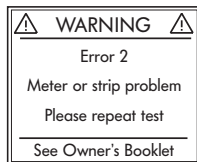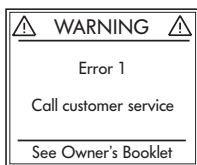

WHAT IT MEANS

There may be a problem with the meter or the meter may need to be reset.



Error message that could be caused by either a used test strip or a problem with the meter.

ACTION

To reset the meter, you must turn the meter off and enter Set-up mode (see pages 58–61). Reset the language, time, and date. You must also reset the meter code to match the code on the test strip vial. If you continue to get an Error 1 message, **Do Not** use the meter. Call **LifeScan Customer Service at 1 800 227-8862** for a replacement meter.

Repeat the test with a new test strip. Refer to pages 7–8 for test strip information. If the error message persists, call **LifeScan Customer Service at 1 800 227-8862.**

MESSAGE

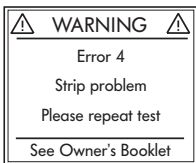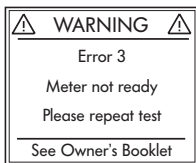

WHAT IT MEANS

Error message that indicates that the blood or control solution sample was applied before “Apply Blood”, “Alternate Site”, or “Control Solution” appeared on the display.

- 1) You may have high glucose and have tested in an environment near the low end of the system’s operating temperature range (43–111°F/ 6–44°C).



ACTION

Repeat the test with a new test strip. Apply the blood or control solution sample only after “Apply Blood”, “Alternate Site”, or “Control Solution” appears on the display. Refer to pages 33–35 for information on sample application.

- 1) If you tested in a cool environment, repeat the test in a warmer environment with a new test strip.

If you were testing in a normal or warm environment, repeat the test with a new test strip (see pages 7–8 for test strip information).

MESSAGE

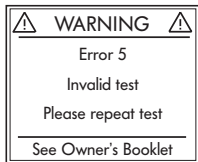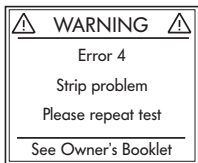




WHAT IT MEANS

- 2) The sample was improperly applied.
- 3) There may be a problem with the test strip. For example, it may have been damaged or moved during testing.
- 1) Error message that indicates the meter has detected a problem with the test strip. Possible causes are test strip damage or an incompletely filled confirmation window.
- 2) There may be a problem with the meter.

ACTION

- 2) If you applied the blood incorrectly, review pages 33–35 on blood application and repeat the test with a new test strip.
- 3) If the error message appears again, call **LifeScan Customer Service at 1 800 227-8862.**
- 1) Repeat the test with a new test strip. Refer to pages 33–35 for information on sample application.
- 2) If the error message appears again, call **LifeScan Customer Service at 1 800 227-8862.**

MESSAGE

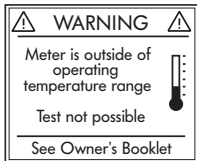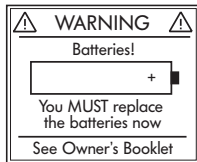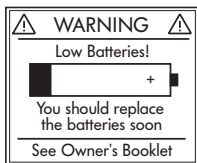

WHAT IT MEANS

The battery sign appears on the display. The power of the batteries is getting low. The backlight will be disabled when the battery sign is displayed. You can complete about 100 more tests from the time this symbol first appears.

Battery power is too low to run a test.

The meter has detected that the temperature is outside the system operating range. **Do Not** perform a test until the meter and test strips reach a temperature within the operating range (43–111°F/6–44°C).

ACTION

Test results will still be accurate, but replace the batteries as soon as possible.

Replace the batteries at once.

Perform the test after the meter and test strips have reached a temperature within the operating range.

PROBLEMS AND POSSIBLE CAUSES

If the meter does not turn on after inserting a test strip or the display is not active:

<u>POSSIBLE CAUSES</u>	<u>WHAT TO DO</u>
No power.	Replace the batteries.
No batteries or batteries incorrectly installed.	Check that the batteries are correctly installed.
Test strip inserted incorrectly. (For example, upside down, backward, or incompletely.)	Insert the test strip correctly with the contact bars end first and facing up. Push it in until it will go no further.
Defective meter or test strip.	Call LifeScan Customer Service at 1 800 227-8862.

If the test does not start after applying the sample:

<u>POSSIBLE CAUSES</u>	<u>WHAT TO DO</u>
Insufficient blood sample.	Repeat the test with a new test strip and a larger sample, making sure blood is placed to the narrow channel in the top edge of the test strip.
Defective test strip.	Repeat the test with a new test strip.
Sample applied after automatic shutoff (three minutes after inserting a test strip).	Repeat the test with a new test strip; apply sample only after “Apply Blood”, “Alternate Site”, or “Control Solution” appears on the display.
Defective meter.	Call LifeScan Customer Service at 1 800 227-8862.

METER SET-UP



Note:

- It is important that the time of day is correct and that your medication set-up and mealtimes are set according to your personal daily schedule. Otherwise, some FastFacts® information will not be correct.
- You do not need to change any of the meter settings in order to begin testing. Simply insert a test strip to turn on the meter and proceed with the test (see pages 19–39).

The OneTouch® UltraSmart® Meter was pre-set at the factory with the date and Pacific Standard Time, and display language. The glucose units are pre-set at the factory and cannot be changed. You must enter Set-up mode to:

- Change the date, time, or display language. (Glucose units cannot be changed.)
- Customize the meter features that allow you to track diabetes pills, insulin doses, and insulin pump information.
- Customize advanced features such as averages, meal schedule, target glucose range, and hypoglycemia level.

Entering Basic Set-up Mode

To enter Set-up mode, press and hold down **BOTH** the Back button  and the OK button  at the same time for at least two seconds.

This message will appear briefly. The language menu will follow immediately. The meter is now in Set-up mode. The SmartButtons® cannot be used while you are in Set-up mode. (You may still test at any time simply by inserting a test strip.)

ONETOUCH
UltraSmart

Selecting Display Language

The meter is set at the factory to use English.

To use English, press the OK button .

Set up in?

English

Español

Français

Deutsch

Italiano

Português

Nederlands

You may choose one of up to six other languages in Set-up mode.

Spanish (Español)



Italian (Italiano)

French (Français)

Portuguese (Português)

German (Deutsch)

Dutch (Nederlands)

To change the language, use the Arrow button  to scroll to the language. With your language choice highlighted, press the OK button .

¿Configurar en?

English

Español

Français

Deutsch

Italiano


Português


Nederlands



Setting Date and Time


After the language has been set, the meter will display the factory-set time and date. If the date and/or time are incorrect, follow these steps in Set-up mode (see page 58, *Entering Basic Set-up Mode*).

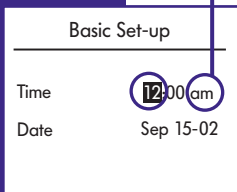

You can ONLY change “am” and “pm” when you pass 12:00. If you use the 12-hour format, make sure “am” or “pm” is correct when setting the hour.



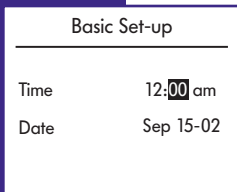

Begin by setting the hour and “am” or “pm” if using 12-hour format. Use the Arrow button  to scroll to the correct hour.

Check am/pm if applicable. If incorrect, continue to press the Arrow button  until the hour and “am” or “pm” are correct. Press the OK button  to continue.




Use the Arrow button  to scroll to the correct minutes and press the OK button .

If “am” or “pm” is not correct, press the Back button  to go back to the previous display and reset to the desired hour and “am” or “pm” setting.



Then select the correct year and press the OK button . Repeat these steps to select the month and day.

Basic Set-up	
Time	12:00 am
Date	Sep 15-02

Note: It is important to select the correct date and time or your logbook may not show the most recent result first.

The meter will ask you if you wish to customize it. Select “Yes” (press the OK button ) if you wish to customize other meter settings (see *Custom Meter Set-up Sequence*, pages 62–76).



Do you want to customize your meter now?	
Yes	No

If you select “No” (use Arrow button ) and press the OK button , the meter will turn off and continue to use the factory settings.

Do you want to customize your meter now?	
Yes	No

CUSTOM METER SET-UP SEQUENCE

The OneTouch® UltraSmart® Meter has a number of optional features that you may use to track all your diabetes information. The Set-up Menu allows you to customize the meter with your personal diabetes management plan including the type and amount of medication you take, your target glucose range, your usual meal times, and much more information.

Note: If you are starting the custom set-up sequence with the meter off, you must press the Back button  and the OK button  at the same time to turn on the meter. To access the Set-up Menu, you must go through *Basic Set-up* first (see pages 58–61).

Set-up Menu

Do you want to customize
your meter now?



After completing Basic Set-up, choose “Yes” to customize your meter.

You should have your OneTouch® UltraSmart® Owner's Booklet available to refer to while customizing the meter.

Please use your Owner's Booklet to help you set up your OneTouch UltraSmart

“Set-up Menu” will appear on the display with a list of options. “Settings” will be highlighted.

Set-up Menu

Settings

Diabetes Pill Set-up
Insulin Set-up
Insulin Pump
Advanced Features
Exit Set-up

Following is a summary of meter settings:

Meter Settings

<u>OPTION</u>	<u>FACTORY SETTINGS</u>	<u>CUSTOM SETTINGS</u>
Settings: Time Format Date Format Begin of Week Cholesterol Units Beep	am/pm MM/DD/YY Sunday mg/dL ON	24-hour DD/MM/YY Monday mmol/L OFF
Diabetes Pill Set-Up:	Pill A Pill B	Pill A to E or Customize with OneTouch® Diabetes Management Software
Insulin Set-Up:	InsulinA InsulinB	Rapid, Regular, Lente, NPH, Ultralen (Ultralente), Premixed, Other, InsulinA, InsulinB, InsulinC, or Customize with OneTouch® Diabetes Management Software




Meter Settings (continued)

<u>OPTION</u>	<u>FACTORY SETTINGS</u>	<u>CUSTOM SETTINGS</u>
Insulin Pump:	No	Yes
Advanced Features:		
Averages	14 days	7, 30, 60, 90 days
Schedule	Pre-set (see pages 74–75)	Personal
Glucose Range Before Meal	90–130 mg/dL	Personal
After Meal	—	Personal
Hypo Level	70 mg/dL	Personal



Settings

“Settings” include:


- Time format
- Date format
- Beginning of the week
- Cholesterol unit of measure
- Beep on or off



To change any of these settings, press the OK button  with “Settings” highlighted on the Set-up Menu. Each setting has a pop-up menu of available choices. You must review all five settings and choose one for each. Press the OK button  to save them. After setting the last setting, the Beep, press the OK button  to save your settings before turning off the meter or allowing it to turn off automatically.

Settings	
Time Format	am/pm
Date Format	24 hour
Begin of week	Sunday
Cholesterol Units	mg/dL
Beep	ON


You may set your meter to use a 12-hour am/pm time format or a 24-hour military time format (24:00). Use the Arrow button  to view the choices. Highlight your choice and press the OK button .

Settings	
Time Format	am/pm
Date Format	MM/DD/YY
Begin of week	DD/MM/YY
Cholesterol Units	mg/dL
Beep	ON



You may choose between the Month-Day-Year date format (e.g., July 5, 2002) or the Day-Month-Year format (e.g., 5 July 2002). Highlight your choice and press the OK button .

For record-keeping purposes you may begin your week on Sunday or Monday. Use the Arrow button  to highlight your choice and press the OK button .

Settings	
Time Format	am/pm
Date Format	MM/DD/YY
Begin of week	Sunday
Cholesterol Units	Monday
Beep	ON

The cholesterol unit of measure may be either mg/dL (used in the U.S. and some European countries) or mmol/L (used in Canada and many other countries). Highlight your choice and press the OK button .

Settings	
Time Format	am/pm
Date Format	MM/DD/YY
Begin of week	Sunday
Cholesterol Units	mg/dL
Beep	mmol/L

The meter was set at the factory to give audio signals (“beeps”) at key points in the test procedure. To keep the beeps, highlight “ON” and press the OK button . For silent meter operation, highlight “OFF” and press the OK button .

Settings	
Time Format	am/pm
Date Format	MM/DD/YY
Begin of week	Sunday
Cholesterol Units	mg/dL
Beep	ON
	OFF

Diabetes Pill Set-up

You may record up to five types of diabetes pills. This feature helps you track when you have taken each of your medications. The names of the medications may be chosen from a menu (Pills A through E) or the brand or generic names entered using a personal computer and OneTouch® Diabetes Management Software. If you do not customize your pill names, the factory settings of Pill A and Pill B will appear on your pill entry display.

Set-up Menu

Settings

Diabetes Pill Set-up

Insulin Set-up

Insulin Pump

Advanced Features


Exit Set-up



Using the Arrow button , scroll to “Diabetes Pill Set-up” and press the OK button .

Diabetes Pill Set-up

How many types of pills do you take?

2


Enter the number of pill **types** you are taking (not the number of tablets you take each time). Scroll to the correct number and press the OK button . You can enter up to five types of pills. If you do not take pills or do not wish to use this feature, enter “0”. If you enter “0”, the diabetes pill choices will not appear on the display when you press the Medication SmartButton®.

A pop-up menu of pill choices will appear on the display. Use Pills A–E to label up to five pills. For example, if you take glipizide pills you could label them as “Pill A”. You can enter the actual pill names using OneTouch® Diabetes Management Software. Using the Arrow button , scroll down to your choice and press the OK button .

Diabetes Pill Set-up

Select Pills



Pill 1:	Pill A
Pill 2:	Pill B
	Pill C
	Pill D
	Pill E

“Pill 2” will be highlighted automatically. To enter a second type of diabetes pill, choose Pill B–E to describe the second pill and press the OK button . Repeat these steps to enter other pill types.

Diabetes Pill Set-up

Select Pills

Pill 1:	Pill B
Pill 2:	Pill C
	Pill D
	Pill E

If you have completed your diabetes pill choices, select “Save” and press the OK button . If necessary, press the Back button  to re-enter pill types.

Diabetes Pill Set-up


Pill 1:	Pill A
Pill 2:	Pill B
Save	

Insulin Set-up

You can record your daily insulin usage by type of insulin and number of units for up to three different insulin types. Insulin types may be chosen from a menu of ten choices. Eleven other insulin types (up to eight characters in length) may be entered using a personal computer loaded with OneTouch® Diabetes Management Software. If you do not customize your insulin names, the factory settings of InsulinA and InsulinB will appear on your medication entry display.

Set-up Menu


Settings
Diabetes Pill Set-up
Insulin Set-up
Insulin Pump
Advanced Features
Exit Set-up

Choose Insulin Set-up on the Set-up Menu and press the OK button .

Insulin Set-up

How many types of insulin do you take?

2

Scroll to the number of insulin types you use and press the OK button . You can choose up to three types of insulin. If you do not wish to use this feature, enter “0”. If you select “0”, no insulin names will appear on the display when you press the Medication SmartButton®.

To enter your first insulin type, review the menu choices:



Rapid	Premixed
Regular	Other
Lente	InsulinA
NPH	InsulinB
Ultralen (Ultralente)	InsulinC

Insulin Set-up

Select insulin(s)

Insulin 1: **Rapid** ▲

Insulin 2: Regular
Lente
NPH
Ultralen ▼



Use the Arrow button  to scroll through the menu. Press the OK button  on your selection. Repeat these steps until all of your insulin types have been selected.

Insulin Set-up

Select insulin(s)

Insulin 1: **Regular** ▲

Insulin 2: Lente
NPH
Ultralen
Premixed ▼

To save your insulin choices, use the Arrow button  to scroll to “Save” and press the OK button .

Insulin Set-up

Insulin 1: Rapid

Insulin 2: Regular


Save

Insulin Pump Set-up

If you are using an insulin pump, the OneTouch® UltraSmart® Meter allows you to store current information about your pump. The factory setting will be set as if you do not use a pump.

Set-up Menu

Settings
Diabetes Pill Set-up
Insulin Set-up
Insulin Pump
Advanced Features
Exit Set-up



Select “Insulin Pump” on the Set-up Menu and press the OK button .

Insulin Pump

Do you use an insulin pump?

Yes

No

Select “Yes” to use this feature and press the OK button . You can then use this feature to enter your bolus doses and total insulin amounts. If you do not use an insulin pump or you do not wish to set up this feature now, select “No” and press the OK button . After choosing “Yes” or “No”, the meter will return you to the Set-up Menu.

Advanced Feature Set-up

The final item on the Set-up Menu is called “Advanced Features”. These features allow you to:

- Choose the number of days included in your glucose test averages
- Input your usual meal schedule
- Include your target glucose range
- Customize your hypoglycemic level

ADVANCED FEATURES

Averages
Schedule

Glucose Range
Before Meal
After Meal

Hypo Level

FACTORY SETTINGS

14 days
Pre-set
(see pages 74–75)

90–130 mg/dL
—

70 mg/dL

CUSTOM SETTINGS


7, 30, 60, 90 days
Personal

Personal
Personal

Personal

Select “Advanced Features” as follows:

Select “Advanced Features” and press the OK button .

The first advanced feature is averages. Each time you test, the meter is set to display a 14-day average of your glucose tests along with your test result. You can change the number of days that make up this average to the last 7, 30, 60, or 90 days. Scroll to the number of days you want to include in your test display average and press the OK button . You can still look at averages for all five time periods by reviewing your electronic logbook.

Set-up Menu

Settings
Diabetes Pill Set-up
Insulin Set-up
Insulin Pump

Advanced Features


Exit Set-up

Advanced Features

Averages	7 Day
Schedule	14
Glucose Range	30
Before Meal	60
After Meal	90
Hypo Level	70


Advanced Features

Averages	14 Day
Schedule	Pre-set
Glucose Range	Personal
Before Meal	90-130
After Meal	--- - ---
Hypo Level	70

Next, the meter will prompt you to select time periods for your usual meal schedule. When you set this feature all of your test results will be tagged with the mealtime. To view a pre-set meal schedule select “Pre-set” and press the OK button .



Pre-set Schedule



Bef Brkft	6:00 a-	9:00 a
Aft Brkft	9:00 a-	11:00 a
Bef Lunch	11:00 a-	2:00 p
Aft Lunch	2:00 p-	5:00 p
Bef Dinner	5:00 p-	8:00 p
Aft Dinner	8:00 p-	11:00 p
Night	11:00 p-	6:00 a

The pre-set schedule will appear on the display. Both Before and After Mealtimes are pre-set. Whenever you test, these mealtimes will be recorded in the electronic logbook whether or not you are eating on schedule. To use the pre-set schedule, press the OK button .

Personal Schedule


Bef Brkft	6:00 a-	9:00 a
Aft Brkft	9:00 a-	11:00 a
Bef Lunch	11:00 a-	2:00 p
Aft Lunch	2:00 p-	5:00 p
Bef Dinner	5:00 p-	8:00 p
Aft Dinner	8:00 p-	11:00 p
Night	11:00 p-	6:00 a

To personalize your schedule, press the Back button , choose “Personal”, and press the OK button . To change all or some of the personal meal schedule:

- Scroll through each mealtime.
- Select the correct hours and minutes (in 15-minute steps).
- Press the OK button .
- If you make a mistake, press the Back button  to return to the top of the schedule and make corrections.

You will only be able to set beginning or start times for each mealtime. End times will change to match the start time of the next time period. For example, if your lunch is usually at 12:00 noon, “Bef Lunch” could be 10:00am to 12:00pm.

Personal Schedule		
Bef Brkft	5:00 a-	8:00 a
Aft Brkft	8:00 a-	10:00 a
Bef Lunch	10:00 a-	12:00 p
Aft Lunch	12:00 p-	5:00 p
Bef Dinner	5:00 p-	8:00 p
Aft Dinner	8:00 p-	11:00 p
Night	11:00 p-	5:00 a

After changing the times, press the OK button  to save your personal schedule.



Personal Schedule		
Bef Brkft	5:00 a-	8:00 a
Aft Brkft	8:00 a-	10:00 a
Bef Lunch	10:00 a-	12:00 p
Aft Lunch	12:00 p-	5:00 p
Bef Dinner	5:00 p-	8:00 p
Aft Dinner	8:00 p-	11:00 p
Night	11:00 p-	5:00 a

Note:

- It is important that the time of day is correct and that your medication set-up and mealtimes are set according to your personal daily schedule. Otherwise, some FastFacts® information will not be correct.
- You have the option to modify the mealtime for any test if necessary.



Advanced Features

Averages	14 Day
Schedule	Pre-set
Glucose Range	
Before Meal	90-130
After Meal	--- - ---
Hypo Level	70

Next, the meter will prompt you to set a before-meal blood glucose range. The meter is pre-set to 90–130 mg/dL. To use this range, press the OK button  twice. You may change this range to one recommended by your healthcare professional. To change the lower and upper ends of the range in 1-mg/dL steps, scroll to the correct numbers and press the OK button .


Advanced Features

Averages	14 Day
Schedule	Pre-set
Glucose Range	
Before Meal	90-130
After Meal	--- - ---
Hypo Level	70

If you have an after-meal glucose range recommended by your healthcare professional and wish to enter it, use the Arrow button  to scroll to your target numbers and press the OK button .


Advanced Features

Averages	14 Day
Schedule	Pre-set
Glucose Range	
Before Meal	90-130
After Meal	--- - ---
Hypo Level	70

The factory-set hypoglycemia level is 70 mg/dL. If your healthcare professional has advised you to use a different level, scroll to the correct number and press the OK button .

Set-up Menu

Settings
Diabetes Pill Set-up
Insulin Set-up
Insulin Pump
Advanced Features
Exit Set-up

To exit Set-up mode, choose “Exit Set-up” and press the OK button .

YOUR ELECTRONIC LOGBOOK

The OneTouch® UltraSmart® electronic logbook is the meter memory. It stores over 3,000 records. The SmartButtons® allow you to enter information about your diabetes management plan, including exercise, health, medication, and food.










Comments

Information that you store in your electronic logbook along with a test result is called a “comment.” Following is a list of comments that you may store.

<u>DEFINITION</u>	<u>COMMENTS ON GLUCOSE RESULTS</u>		
Exercise	Before	During	After
Health	Stress Menses	Feel Hypo Vacation	Illness Other
Medication	None		
Food	Before Breakfast Before Lunch Before Dinner Night	After Breakfast After Lunch After Dinner	

Rules for Adding Comments

Following are rules for adding comments in your logbook.


- If you wish to add a comment to a test result, press the OK button  while viewing the test result.
- Use the Arrow button  to scroll up and down through the various menus and choices.
- Use the OK button  to make a selection.
- You must save your selections by using the Arrow button  to highlight “Save” and pressing the OK button .
- Prior to seeing the logbook display, you may press the Back button  and return to the previous display to review or edit the information you have stored.
- To remove an entry, select  from the menu.
- Press and hold the Back button  to turn off the meter.
- After one minute of no activity, the meter will turn off automatically. To go back into the logbook, press the FastFacts® SmartButton®. With “Logbook” highlighted, press the OK button .

You can comment on a test as occurring before or after a meal, how you are feeling at the time of the test, and if there was any exercise involved. In all, you may add one food comment, one exercise comment, and up to six health comments to a test result.

How to Add a Comment





Following are the steps you take to comment on a test result.

The meter will automatically prompt you to add a comment when you have a test result outside of your glucose target range. You can, however, add a comment to a test result whether or not “Comment? Press OK” appears. In this example, we will comment on a test result with food, exercise, and health information.

With any test result (or test result with average) on the display, press the OK button . You will be able to add comments for food, health, and exercise.

Health is always highlighted first since the food comment is automatically added based on your pre-set or personal schedule (see *Advanced Feature Set-up*, pages 72–76).

FOOD COMMENTS

In this example we'll use food. If you wish to change the food comment to a different mealtime, use the Arrow button  to select “Food” and press the OK button . Use the Arrow button  to select your food comment and press the OK button  again. Your choices are Bef Brkft, Aft Brkft, Bef Lunch, Aft Lunch, Bef Dinner, Aft Dinner, and Night.

Sep 8-02 10:30 am

135

mg/dL

Comment? Press OK 

(Example)

Sep 8-02 10:30 am

Gluc. 135 Comment

Food Aft Brkft

Health -----


Exer -----

Save

Sep 8-02 10:30 am

Gluc. 135 Comment

Food

Health ----- 


Exer

Save

Bef Brkft



Aft Brkft

Bef Lunch

Aft Lunch 

Sep 8-02 10:30 am
Gluc. 135 Comment

Food Bef Lunch
Health -----
Exer -----
Save


Once you select your mealtime, “Health” is automatically highlighted. If you are through commenting, scroll to “Save” and press the OK button . **If you do not press the OK button , your entries will not be saved.**

HEALTH COMMENTS

Sep 8-02 10:30 am
Gluc. 135 Comment

Food
Health
Exer
Save

----- ▲
Stress
Feel Hypo
Illness
Menses ▼


You can comment on the result with notes about your overall health at the time of the test. Press the OK button  with “Health” highlighted. You may add up to six comments from this menu:



Stress	Menses
Feel Hypo	Vacation
Illness	Other

Sep 8-02 10:30 am
Gluc. 135 Comment

Food
Health 1
Health 2
Exer
Save


----- ▲
Feel Hypo
Illness
Menses
Vacation ▼

After adding your first health comment, “Health 2” will automatically appear on the display. To add additional health comments, press the OK button  and select the comment from the menu. Note that your previous comment is no longer available.

When you have completed making comments, scroll to “Save” and press the OK button . **If you do not press the OK button , your entries will not be saved.**

EXERCISE COMMENTS

You can comment on a test result as occurring before, during, or after exercise.

After you add the exercise comment, “Save” will automatically be highlighted. If you press the OK button , all of the comments you have entered will appear with the test result in the logbook.

Sep 8-02 10:30 am Gluc. 135 Comment	
Food	Bef Lunch
Health 1	Stress
Health 2	-----
Exer	-----
Save	

Sep 8-02 10:30 am Gluc. 135 Comment	
Food	Bef Lunch
Health 1	-----
Health 2	Before
Exer	During
Save	After

Sep 8-02 10:30 am Gluc. 135 Comment	
Food	Bef Lunch
Health 1	Stress
Health 2	-----
Exer	After
Save	

Logbook Entries

“Logbook entries” are activities concerning exercise, health, medication, and food that are not associated with a test. **You do not have to perform a blood glucose test in order to add valuable diabetes information to your electronic logbook.**

Using the SmartButtons®

The SmartButtons® allow you to enter information about your diabetes management plan. There are SmartButtons® for exercise, health, medication, and food. To make a logbook entry, press the appropriate SmartButton® to turn on the meter and enter the menu for the type of entry.

Note: You may press any SmartButton® to turn on the meter and move directly to the category.



Following is a list of logbook entries you can make.

SmartButton®

LOGBOOK ENTRIES

Exercise



Mild
Moderate
Hard

Duration (enter
length of time)

Health



Health Notes:

Stress
Feel Hypo
Illness

Menses
Vacation
Other

Health Checks:

Ketones
HbA1c
Microalbumin
Cholesterol
Blood Pressure

Eye Exam
Foot Exam
Weight/Height
Dr. Visit

Medication



Pills (1–5 and how many of each)
Insulin (1–3 and units)
Insulin Pump







Food




Carbs
Calories

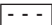
Fats
Proteins

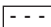Rules for Logbook Entries

- If you wish to add a logbook entry, press a SmartButton® to turn on the meter. The icon for the category you are choosing will appear in the upper-left-hand corner of the display and the title of the category will appear at the top of the display. If you press a SmartButton® in error, either press another SmartButton® or choose  from the selection menu.
- Before making a logbook entry, the time must be selected. Use the Arrow button  to choose either the date and time displayed or “Other Time”.
- Use the OK button  to confirm your choices. The logbook display will appear. If no logbook display appears, your entries have not been saved.
- It is possible to enter the same logbook entry information more than once for a given date and time.
- To remove an entry, select  from the menu.
- Prior to seeing the logbook display, you may press the Back button  and return to the previous display to review or edit the information you have stored.
- Press and hold the Back button  to turn off the meter.

Entering the Date and Time

If the logbook entry is for the current date and time, press the OK button .

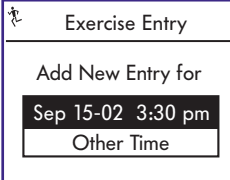


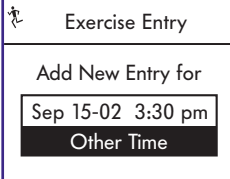
Exercise Entry

Add New Entry for

Sep 15-02 3:30 pm

Other Time

If the entry is for a previous date and time, select “Other Time” and press the OK button .




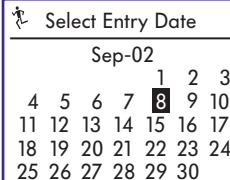
Exercise Entry

Add New Entry for

Sep 15-02 3:30 pm

Other Time


A calendar will appear on the display. Scroll to the correct month and day and press the OK button .




Select Entry Date

Sep-02

				1	2	3			
4	5	6	7	8	9	10			
11	12	13	14	15	16	17			
18	19	20	21	22	23	24			
25	26	27	28	29	30				

Record the time in hours and minutes and press the OK button . Note that “am” and “pm” change every 12 hours. Then make your logbook entry.

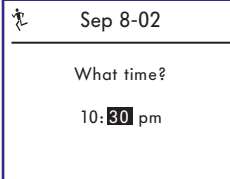


Sep 8-02

What time?

10:30 pm


EXERCISE LOGBOOK ENTRIES


 Exercise Entry

Add New Entry for

Sep 15-02 3:30 pm

Other Time

With the meter turned off, press the Exercise SmartButton®. These choices will appear on the display. Select the current date and time to add an exercise logbook entry at this time. Select “Other Time” if the logbook entry is for some time in the past and press the OK button .


 Sep 15-02 3:30 pm
Exercise Entry


Exercise -----

Duration Mild

Moderate


Hard


Note: Check that the date and time are correct for your entry. You may rate the exercise you performed as mild, moderate, or hard. Press the OK button .


 Sep 15-02 3:30 pm
Exercise Entry

Exercise Hard

Duration 00:10


Then record the duration of the exercise to the nearest five minutes. Press the OK button  to save the logbook entry.


The exercise logbook entry you recorded will be saved this way in your electronic logbook. You may also scroll through the logbook to view previous entries. Press the OK button  while you are in the logbook to edit or delete the highlighted entry (see pages 105–107).

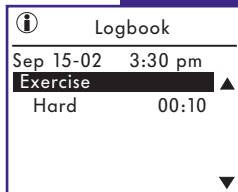

HEALTH LOGBOOK ENTRIES


After selecting the date and time, two menu choices will appear. You may make health logbook entries for the current time or another time (see page 85). “Health Notes” allow you to enter information about your day.


Select “Health Notes” and press the OK button .

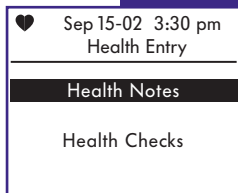

Choose from this menu:

Stress	Menses
Feel Hypo	Vacation
Illness	Other

Press the OK button  to save and view the logbook entry.

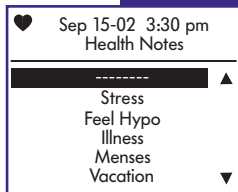

Logbook

Sep 15-02 3:30 pm

Stress ▲

▼

The logbook entry with your Health Note will be saved this way in the electronic logbook.


“Health Checks” are another type of health logbook entry that you can record. Health Checks include home and laboratory tests as well as doctors’ examinations. All of this information is vital to a good diabetes management plan. Health Checks may be stored in your electronic logbook at the time they occur or at some later time.

♥ Sep 15-02 3:30 pm

Health Entry

Health Notes

Health Checks

To record check-up information, select “Health Checks” and press the OK button .

Choose from this menu:

Ketones

HbA1c

Microalbumin

Cholesterol




Blood Pressure

Eye Exam

Foot Exam

Weight/Height

Dr. Visit

Many Health Checks allow you to input numeric values. In order to adjust the number, use the Arrow button  to scroll up and down. Press the OK button  to save your selection and return to the Health Checks menu. Following are sample entries possible under “Health Checks”. When you have completed your Health Check entries, scroll to “Exit Health Checks” and press the OK button . Your Health Check entry will be saved in the electronic logbook.



Ketones

When you check ketones, you can enter the results in your electronic logbook.




Results may be:

Negative	Moderate
Trace	Large
Small	

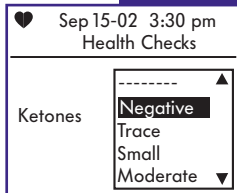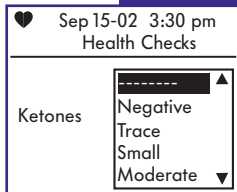Scroll to your selection and press the OK button .




♥	Sep 15-02 3:30 pm	
Health Checks		
Blood Pressure		▲
Eye Exam		
Foot Exam		
Weight/ Height		
Dr. Visit		
Exit Health Checks		▼


You will return to the Health Checks menu with “Exit Health Checks” highlighted. If you are finished with your entries, press the OK button  with “Exit Health Checks” highlighted. If you wish to enter other Health Checks, use the Arrow button  to scroll to the additional menu choice and press the OK button .

ⓘ	Logbook	
Sep 15-02	3:30 pm	
Ketones	Negative	▲
		▼

Your entry will be saved this way in the electronic logbook.


HbA1c


Tests for hemoglobin A1c, also known as HbA1c or A1c, can be recorded from 4% to 15% in 0.1% steps. When entering data, the value will start at the value you last saved. Scroll to your selection and press the OK button .


 Sep 15-02 3:30 pm
Health Checks

HbA1c **5.2%**

Microalbumin

You can enter your test results for microalbumin as “Normal”, “Positive”, a numeric value from 1 to 1000 or “>1000”. When entering data, the value will start at the value you last saved. Scroll to your selection and press the OK button .

 Sep 15-02 3:30 pm
Health Checks

Microalbumin


Cholesterol



♥ Sep 15-02 3:30 pm
Health Checks

Cholesterol

Total: 187
LDL: ----
HDL: ----
Trig: ----

You may enter cholesterol test results in 1-mg/dL (0.1-mmol/L) steps for:

- Total (up to >1000) (>25.9 mmol/L)
- LDL (up to >500) (>13.0 mmol/L)
- HDL (up to >500) (>13.0 mmol/L)
- Triglycerides (Trig) (up to >3000) (>33.9 mmol/L)

To advance to the next cholesterol entry, press the OK button . To return to a previous cholesterol entry on this display, press the Back button .

Note: When entering data, the value will start at the value you last saved.

Blood Pressure


The logbook range for the first number (systolic) is 80–200 mmHg. The range for the second number (diastolic) is 40–150 mmHg. Both are in 1-mmHg steps. Blood pressure (B.P.) above or below these ranges will appear on the display as “- - -” and will not be recorded in the logbook.


♥ Sep 15-02 3:30 pm
Health Checks

Blood Pressure
120/ 80

When entering data, the value will start at the value you last saved.


Eye Exam


To record an eye examination, select “Enter Eye Exam” and press the OK button .

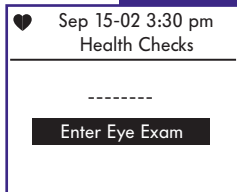

The screenshot shows a mobile application interface. At the top, there is a status bar with a heart icon, the date and time 'Sep 15-02 3:30 pm', and the text 'Health Checks'. Below this is a horizontal line. Under the line, there is a dashed line '-----'. At the bottom, there is a black button with the text 'Enter Eye Exam' in white.


Foot Exam

To record a foot examination, select “Enter Foot Exam” and press the OK button .

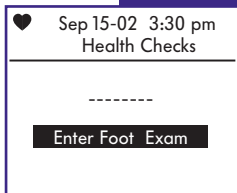

The screenshot shows a mobile application interface. At the top, there is a status bar with a heart icon, the date and time 'Sep 15-02 3:30 pm', and the text 'Health Checks'. Below this is a horizontal line. Under the line, there is a dashed line '-----'. At the bottom, there is a black button with the text 'Enter Foot Exam' in white.


Weight/Height

 Sep 15-02 3:30 pm
Health Checks

Weight	150.0
Height	67.0

Enter your weight in 0.5-pound steps and your height in 0.5-inch steps. When entering data, the value will start at the value you last saved.

Doctor Visit


 Sep 15-02 3:30 pm
Health Checks

Enter Dr. Visit

To record the date of your last visit to the doctor, select “Enter Dr. Visit”.

MEDICATION LOGBOOK ENTRIES

Press the Medication SmartButton® to access the menu. You may track your diabetes pills, insulin types and dosages, and insulin pump information. Select the current date and time or “Other Time” (see page 85).

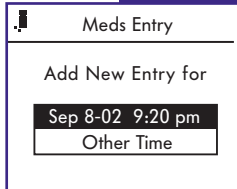

Meds Entry

Add New Entry for

Sep 8-02 9:20 pm


Other Time

If you entered “0” pills, “0” insulin, and “No” for pump in the Set-up mode, this message will appear. To select your medications return to Set-up mode (see *Custom Meter Set-up Sequence*, pages 62–76).

Please use Set-up mode to select pills, insulin or pump functions.

Medication Menu

The medication menu reflects your input in the Set-up mode (see pages 68–72). If you entered diabetes pills, insulin, and/or insulin pump information, it will appear on the display as in this example.

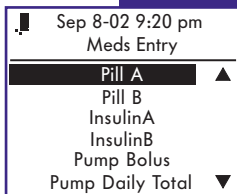

Sep 8-02 9:20 pm
Meds Entry

Pill A ▲

Pill B

InsulinA

InsulinB

Pump Bolus

Pump Daily Total ▼

Diabetes Pills

Sep 8-02 9:20 pm
Meds Entry

Pill A ▲


Pill B

InsulinA

InsulinB


Pump Bolus

Pump Daily Total ▼

To enter your diabetes pills, select the pill you have taken and press the OK button .

Sep 8-02 9:20 pm
Meds Entry

Pill A 1.0

Enter the number of pills you have taken (in 0.5 or one-half pill steps) and press the OK button . When entering data, the value will start at the value you last saved.

Sep 8-02 9:20 pm
Meds Entry

Pill B ▲


InsulinA


InsulinB

Pump Bolus

Pump Daily Total

Exit Meds Entry ▼

After your medication logbook entry, select “Exit Meds Entry” and press the OK button  or select another item from the menu.

 Logbook


Sep 8-02 9:20 pm


Pill A 1.0 ▲

▼


The logbook entry will be saved this way in your electronic logbook.


Insulin Dosages


Enter an insulin dosage by highlighting an insulin type and press the OK button .

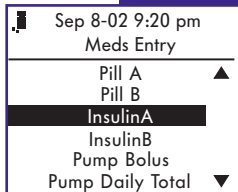


Then record the units of insulin. The meter was set at the factory to record units in 0.1 increments. When entering data, the value will start at the value you last saved.

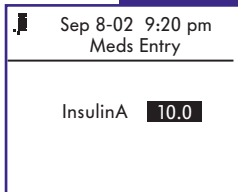

If you wish to record a second insulin, choose from the list, and then enter the number of units. Repeat these steps as needed. After completing your insulin logbook entry, select “Exit Meds Entry” and press the OK button .

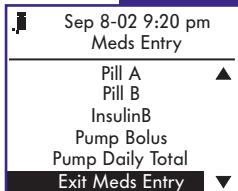


The insulin dosage will be saved this way in your electronic logbook.

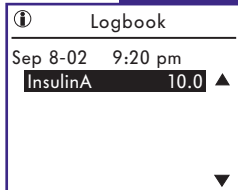


Insulin Pump


You can enter insulin pump information as bolus entries or as a daily total.

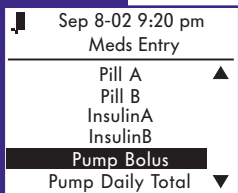

Sep 8-02 9:20 pm
Meds Entry


Pill A ▲
Pill B
InsulinA
InsulinB
Pump Bolus
Pump Daily Total ▼

If you are entering bolus information, select “Pump Bolus” and press the OK button .

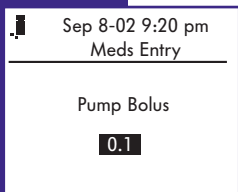


Sep 8-02 9:20 pm
Meds Entry

Pump Bolus
0.1

Then enter the units in 0.1 steps and press the OK button . When entering data, the value will start at the value you last saved.


If you have completed your medication logbook entry, select “Exit Meds Entry”.

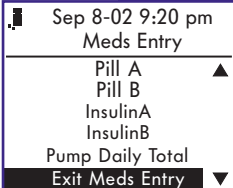

Sep 8-02 9:20 pm
Meds Entry

Pill A	▲
Pill B	
InsulinA	
InsulinB	
Pump Daily Total	
Exit Meds Entry	▼

The dosage will be saved this way in your electronic logbook.

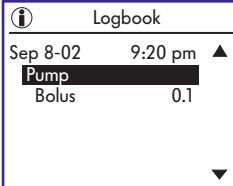


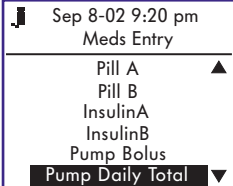
Logbook

Sep 8-02 9:20 pm ▲

Pump	
Bolus	0.1

▼

To enter your pump total for the day, scroll to “Pump Daily Total” and press the OK button .





Sep 8-02 9:20 pm
Meds Entry

Pill A	▲
Pill B	
InsulinA	
InsulinB	
Pump Bolus	
Pump Daily Total	▼

Sep 8-02 9:20 pm
Meds Entry

Pump Daily Total

25.2

Use the Arrow button  to scroll to the number of units you have taken over the past 24 hours and press the OK button . When entering data, the value will start at the value you last saved.

Sep 8-02 9:20 pm
Meds Entry

Pill A ▲

Pill B


InsulinA

InsulinB

Pump Bolus

Exit Meds Entry ▼

When you have completed your medication logbook entries, select “Exit Meds Entry”.

 Logbook

Sep 8-02 9:20 pm ▲

Pump

Daily Total 25.2

▼

Your daily pump total will be saved this way in your electronic logbook.

FOOD LOGBOOK ENTRIES

Food and blood glucose levels are closely linked. Logbook entries allow you to keep track of important nutritional information for each meal or snack by type of nutrient.

Press the Food SmartButton® and select either the current date and time or “Other Time”.



ψ	Food Entry
Add New Entry for	
Sep 8-02 5:26 pm	
Other Time	

Then enter the type of meal from the meal menu. You may also record alcohol usage.



ψ	Sep 8-02 5:26 pm
Food Entry	

Breakfast	
Lunch	
Dinner	
Snack	
Alcohol	

Dinner Information	
Carbs:	---
Fats:	---
Calories:	---
Proteins:	---

You may enter carbohydrates (carbs), fats, and proteins in the range of 0 to 250 in 1-unit steps. Carbohydrates may be entered as units or grams, whichever you prefer. Calories may be entered in 5-unit steps in the range of 0 to 2550. Use the Arrow button  to change the units for each item. If you do not wish to enter units for an item, select “- - -” and press the OK button . **Do Not enter “0” because it will be included in your FastFacts® averages (see page 108).**

Dinner Information	
Carbs:	60
Fats:	15
Calories:	375
Proteins:	-----



Press the OK button  to advance to the next nutrient. To return to a previous nutrient, press the Back button .

Note: When entering data, the value will start at the value you last saved.


Logbook	
Sep 8-02 5:26 pm	
Dinner ▲	
Carbs:	60
Fats:	15
Calories:	375
▼	


Your food logbook entry will be saved this way in the electronic logbook.

Adding, Changing, or Deleting Logbook Information


- Step 1. Press the FastFacts® SmartButton® to enter the FastFacts® menu.
- Step 2. Select “Logbook” from the FastFacts® menu and press the OK button . The most recent entry in your electronic logbook will appear on the display.
- Step 3. Use the Arrow button  to scroll through your logbook entries.


Note the location of the highlight in the logbook display. Note also on the right-hand margin the arrows (▲▼). They indicate the direction you may scroll through your logbook.

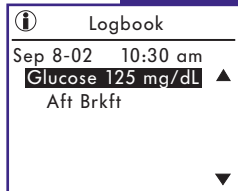


- Step 4. Press the OK button  to edit (add to or change) or delete logbook information.

EDITING OR DELETING TEST COMMENTS (Example)

To edit or delete a comment, be sure the highlight is on the glucose test result to which the comment is attached. Press the OK button  (see page 105 for electronic logbook entry example).

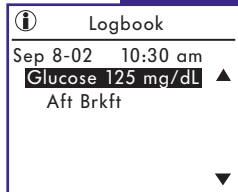



Sep 8-02 10:30 am Gluc. 125 Comment	
Food	Aft Brkft
Health	-----
Exer	-----
Save	

To change the mealtime, select “Food” and press the OK button .


Sep 8-02 10:30 am Gluc. 125 Comment	
Food	----- ▲
Health	Bef Brkft
Exer	Aft Brkft
Save	Bef Lunch
	Aft Lunch ▼

Scroll to the desired mealtime and press the OK button  again.

Sep 8-02 10:30 am Gluc. 125 Comment	
Food	Bef Lunch
Health	-----
Exer	-----
Save	



You may edit other comments at this time. To delete a comment, scroll to “- - -” and press the OK button . When you have completed editing the comments, scroll to “Save” and press the OK button .

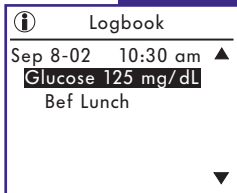
Your edited comment will appear this way in your electronic logbook.



A screenshot of a mobile application titled "Logbook". At the top, there is an information icon (i) and the title "Logbook". Below this, the date and time "Sep 8-02 10:30 am" are displayed, followed by a small upward-pointing triangle. The main entry is "Glucose 125 mg/dL", which is highlighted with a black background. Below the entry is the text "Bef Lunch". At the bottom right, there is a downward-pointing triangle.


EDITING OR DELETING ELECTRONIC LOGBOOK ENTRY (Example)

All electronic logbook entries are edited or deleted in the same way. Use the Arrow button  to highlight the electronic logbook entry you wish to change and press the OK button .

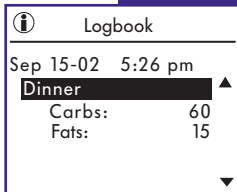

A screenshot of the "Logbook" app. The title "Logbook" is at the top. Below it, the date and time "Sep 15-02 5:26 pm" are shown with an upward-pointing triangle. The entry "Dinner" is highlighted with a black background. Below "Dinner", the text "Carbs: 60" and "Fats: 15" is displayed. A downward-pointing triangle is at the bottom right.




Press the OK button  to edit.



A screenshot of the "Logbook" app showing a menu. At the top, there is a fork and knife icon, followed by the date and time "Sep 15-02 5:26 pm" and the text "Food Entry". Below this, there are two buttons: "Edit" and "Delete", each in its own box.


ψ	Sep 15-02 5:26 pm
Food Entry	

Breakfast	
Lunch	
Dinner	
Snack	
Alcohol	

Make sure the icon, date and time, and description are correct. (If you make an error, press the Back button  to return to the menu.) Press the OK button  on the selected mealtime. If you wish to change the entry to another mealtime, scroll to the mealtime and press the OK button .



ψ	Dinner Information

Carbs:	60
Fats:	15
Calories:	---
Proteins:	---


To keep the entry as it is, press the OK button .

ψ	Dinner Information


Carbs:	45
Fats:	15
Calories:	---
Proteins:	---


To change the entry, use the Arrow button  and press the OK button .

The edited entry will appear this way in your electronic logbook.

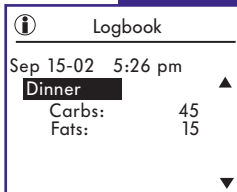




The screenshot shows the 'Logbook' app interface. At the top, there is an information icon and the title 'Logbook'. Below this, the date and time 'Sep 15-02 5:26 pm' are displayed. The main entry is 'Dinner', which is highlighted with a black background. To the right of 'Dinner', there are two sub-entries: 'Carbs: 45' and 'Fats: 15'. On the far right, there are upward and downward arrow icons. The entire interface is enclosed in a purple border.

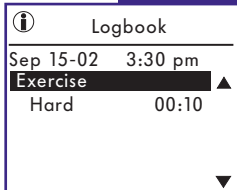To delete an entry, highlight the entry and press the OK button .




The screenshot shows the 'Logbook' app interface. At the top, there is an information icon and the title 'Logbook'. Below this, the date and time 'Sep 15-02 3:30 pm' are displayed. The main entry is 'Exercise', which is highlighted with a black background. To the right of 'Exercise', there is a sub-entry 'Hard 00:10'. On the far right, there are upward and downward arrow icons. The entire interface is enclosed in a purple border.

Use the Arrow button  to select "Delete" and press the OK button .

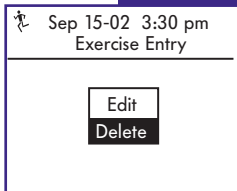

The screenshot shows the 'Logbook' app interface. At the top, there is a person icon, the date and time 'Sep 15-02 3:30 pm', and the title 'Exercise Entry'. Below this, there are two buttons: 'Edit' and 'Delete'. The 'Delete' button is highlighted with a black background. The entire interface is enclosed in a purple border.

Your entry will be deleted from the electronic logbook.

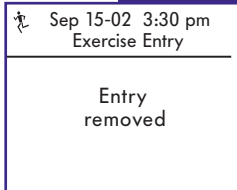

The screenshot shows the 'Logbook' app interface. At the top, there is a person icon, the date and time 'Sep 15-02 3:30 pm', and the title 'Exercise Entry'. Below this, the text 'Entry removed' is displayed. The entire interface is enclosed in a purple border.

OneTouch® UltraSmart® FastFacts® FEATURE


FastFacts® Menu



Enter FastFacts® by pressing the FastFacts® SmartButton®. This will put you in the meter memory. The FastFacts® feature allows you to view charts and graphs of your test results and other logbook information you have entered.

FastFacts® SmartButton®

Note: The meter will turn itself off after one minute of inactivity. Press any SmartButton® to turn the meter back on.

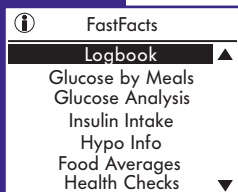


The FastFacts® menu lists several choices. See the next page for a description of what is available under each choice.

FastFacts® Menu

Logbook (Meter Memory)	■ Glucose test results with comments and logbook entries by date and time
Glucose by Meals	■ Glucose test results by date before and after meals
Glucose Analysis	<ul style="list-style-type: none">■ Graph of All Results. Interactive graph of all glucose test results by date■ Graph by Time of Day. Graph of glucose results by time of day■ Average of All Results. Average of all results taken for the last 7, 14, 30, 60, and 90 days■ Average by Time of Day. Averages by time of day for the last 7, 14, 30, 60, and 90 days■ Averages by Exercise. Glucose averages before, during, and after exercise for the last 7, 14, 30, 60, and 90 days■ Glucose Range Info. Percent of glucose test results within, above, and below target range, before and after meals, for the last 7, 14, 30, 60, and 90 days
Insulin Intake	■ Average insulin doses by insulin type, before and after meals, for the last 7, 14, 30, 60, and 90 days
Hypo Info	■ Incidents of hypoglycemia (blood glucose of 70 mg/dL or less) before and after meals for the last 7, 14, 30, 60, and 90 days
Food Averages	■ Average daily intake of carbohydrates, fats, calories, and proteins for the last 7, 14, 30, 60, and 90 days
Health Checks	■ Dates and results of medical tests
Help	■ LifeScan Web site address

LOGBOOK MEMORY

To review the details of each glucose test, comments, and logbook entries, select “Logbook”. The logbook is the meter memory. You may review all entries in your electronic logbook, by date and time, by pressing the Arrow button . In addition to reviewing records, you may also edit the records (see pages 103-107). Following are sample records. Comments are attached to a test result; logbook entries are not.



Logbook	
Sep 8-02	10:30 am
Glucose 125 mg/dL ▲	
Bef Lunch	
Before Exer	

Test Result with Comments

Logbook	
Sep 15-02	3:30 pm
Exercise ▲	
Hard	00:10

Logbook Entry



GLUCOSE BY MEALS


To review test results before and after meals, select “Glucose by Meals”. Highlight the desired mealtime and press the OK button . A summary of all test results by date before and after the chosen meal or at night will appear. Use the Arrow button  to view more entries.

Glucose by Meals	
Breakfast	
Lunch	
Dinner	
Night	

Glucose by Meals		
	Bef Brkft	Aft Brkft
Sep 8	80	120
Sep 7	100	115
Sep 6	70	125
Sep 6	140	---
Sep 5	70	125
Sep 4	90	140 ▼



GLUCOSE ANALYSIS

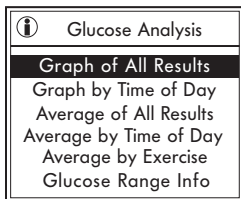


When you select “Glucose Analysis” on the FastFacts® menu, an additional menu of choices appears. Use the Arrow button  to move to the desired menu and press the OK button .



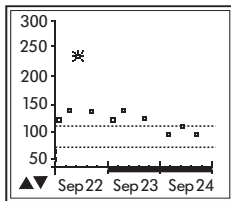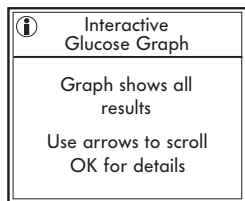


Interactive Graph of All Results

The interactive graph displays three days of test results at a time with the most recent test result flashing. A bold bar above the dates indicates a weekend. The two dotted lines indicate the lowest and highest values of your before- and after-meal glucose ranges (see *Advanced Feature Set-up*, pages 72–76). If you have not selected an after-meal range, only the before-meal range will appear. Results above 300 mg/dL or below 50 mg/dL are indicated by an arrow at the top or bottom edge of the graph.




You may scroll backward or forward in time using the Arrow button  to move from one point to another. Individual test results will flash as you scroll. To view the details of a test, press the OK button  while that test result is flashing. Press the Back button  to return to the graph. You may move back and forth between the graph and the logbook as often as you wish.

Note: If you press the OK button  while in the logbook, you will begin to edit or delete test comments and will not be able to use the Back button  to return to the graph.


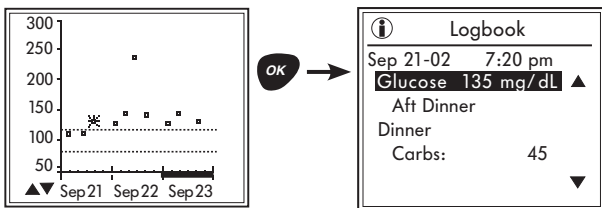


Graph by Time of Day

You may view test results on a graph by time period when you select “Graph by Time of Day”.

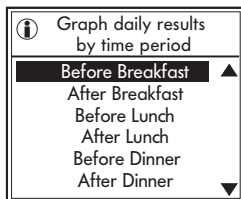

Results by time period are shown day by day, one week at a time. Each time you reach the end of the graph, it will move by one day. Weekends are noted by a bold bar above the dates.

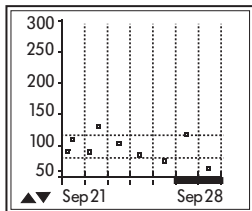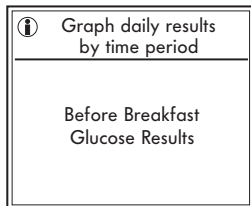

Average of All Results

Test result averages are available for the last 7, 14, 30, 60, and 90 days. The number of tests that make up each average is shown on the left in parentheses.

i Average of all Results		
	Number of Days	Avg
(18)	7	120
(33)	14	147
(75)	30	187
(153)	60	200
(202)	90	201

Average by Time of Day

Test averages by time of day are available for the last 7, 14, 30, 60, and 90 days. The number of tests that make up each average is shown on the left in parentheses.

i	Average by time of Day
7	Days
14	
30	
60	
90	

i	Avg. by time of Day - 14 Days
(14)	Bef Brkft 95
(14)	Aft Brkft 135
(14)	Bef Lunch 80
(14)	Aft Lunch 128
(14)	Bef Dinner 128
(7)	Aft Dinner 125 ▼

Average by Exercise


Test result averages before, during, and after exercise are available for the last 7, 14, 30, 60, and 90 days. The number of tests that make up each average is shown on the left in parentheses.


i	Average by Exercise
7	Days
14	
30	
60	
90	


i	Avg. by Exercise 14 Days
(14)	Before exercise 125
(3)	During exercise 103
(14)	After exercise 110


Glucose Range Info

If you entered your before- and after-meal glucose ranges in the Set-up mode, you may review the percentage of your test results that are above, below, and within your target ranges. The number of tests that make up each percentage is shown on the left in parentheses. Otherwise, before-meal ranges are factory-set at 90–130 mg/dL. After-meal target ranges must be set by the user (see pages 75–76).

 Glucose Range Info
Before Meal
After Meal


 Bef Meal Glucose Range 90-130
Breakfast
Lunch
Dinner
Night
Total

 Bef Brkft Glucose Range 90-130
<div><div>7</div><div>14</div><div>30</div><div>60</div><div>90</div></div> Days

 Bef Brkft Glucose Range 90-130
(6) Above 20%
(5) In Range 70%
(3) Below 10%

INSULIN INTAKE

Choose “Insulin Intake” from the FastFacts® Menu.



 Insulin Intake


Regular

NPH

Syringe Totals



Pump


The individual insulins, in this example Regular and NPH, are based on insulins selected in the Set-up mode (see pages 70–71). Use the Arrow button  to move to your selection and press the OK button .

 Insulin Intake
Regular

7
14
30
60
90

Days

Choose the number of days you would like to review in your average by using the Arrow button  to scroll through choices. Press the OK button  to proceed.

 Regular Insulin
14 Day (Avg.)


(14) Breakfast 8.0 ▲
6:00am - 11:00am


(14) Lunch 10.5
11:00am - 5:00pm ▼

In this example, the average intake of Regular insulin over the past 14 days is displayed. The number in parentheses before the time segment is the number of logbook entries for that time period.


Note: Each time period includes the before and after period for that meal (see pages 74–75 to change your schedule).

Syringe Totals


To view the total of all individual insulins taken by syringe or insulin pen that you have previously stored in the logbook, select “Syringe Totals” and press the OK button .


	Insulin Intake
<hr/>	
Regular	
NPH	
Syringe Totals	
Pump	


You may view the daily average of your insulin intake over the last 7, 14, 30, 60, and 90 days (example 14 days).


	Insulin Intake Syringe Totals
<hr/>	
Daily Average	
Daily Totals	

In this example, the average number of insulin units taken over the last 14 days appears. The number shown in parentheses is the number of syringe insulin entries over the time period. An asterisk (*) after the number means there has been at least one day in the time period without an entry.

	Insulin Intake Syringe Totals
<hr/>	
14 Day (Avg.) Total Daily Insulin	
24.6	
(18)*	


	Insulin Intake Syringe Totals
Daily Average	
Daily Totals	

To view your total syringe insulin intake for each day, scroll to “Daily Totals” and press the OK button .

	Insulin Intake Daily Totals
Sep 15 - 02	85.0
Sep 14 - 02	92.0
-----	-----
Sep 12 - 02	76.5
Sep 11 - 02	56.0
Sep 10 - 02	84.0 ▼


If you did not record a syringe insulin intake for a particular day, “- - -” will appear in both columns.

Insulin Pump


	Insulin Intake
Premixed	
Rapid	
Syringe Totals	
Pump	

Select “Pump” to view bolus intake, daily averages, and daily totals. You can review the last 7, 14, 30, 60, or 90 days.


“Bolus” will display your average bolus intake for each mealtime (Breakfast, Lunch, Dinner, Night) based on your schedule (see pages 73–76).

	Insulin Intake Pump
<hr/>	
Bolus	
<hr/>	
Daily Average	
Daily Totals	

This example shows the average bolus intake in units by time. The number of logbook entries appears in parentheses.

	Pump Bolus 14 Day (Avg.)
<hr/>	
(14)	Breakfast 6.0 ▲ 6:00am - 11:00am
(14)	Lunch 8.0 11:00am - 5:00pm
▼	

“Daily Average” displays the average of the total daily pump insulin. This example is for the last 14 days. The number of entries is listed in parentheses. An asterisk (*) means there are days with no entries.


	Insulin Intake Pump
<hr/>	
14 Day (Avg.)	
Total Daily Pump Insulin	
(12)*	52.3

Insulin Intake Pump Daily Totals

“Pump Daily Totals” gives the pump totals you have entered for each day. If you did not enter a pump total for a day, “- - -” will appear.

Insulin Intake Pump Daily Totals	
Sep 14-02	85.0
Sep 14-02	92.0
Sep 13-02	82.5
Sep 12-02	76.5
-----	-----
Sep 10-02	84.0 ▼

HYPOGLYCEMIA INFORMATION

If you select “Hypo Info” from the FastFacts® menu, the meter will display the actual number of hypoglycemic events defined by the level set in Advanced Features (see page 76). The number of events before and after meals and during the night for the last 7, 14, 30, 60, and 90 days is displayed. Use the Arrow button  to view the other time periods.

Hypo Info (70 mg/dL or less)	
7	Days
14	
30	
60	
90	

Hypo Info (70 mg/dL or less)	
Total - 14 Days:	6
Bef Brkft	5
Aft Brkft	1
Bef Lunch	0
Aft Lunch	0 ▼



FOOD AVERAGES

Select “Food Averages” to view average daily intake of carbohydrates, fats, calories, and proteins over the last 7, 14, 30, 60, and 90 days. The number of tests that make up each average is shown on the left in parentheses.

Food Averages	
7	Days
14	
30	
60	
90	

Food Averages	
Breakfast	
14 Day (Avg.)	
(14) Carbs:	45
(14) Fats:	14
(10) Calories:	362
(12) Proteins:	14

HEALTH CHECKS

To view results and dates of various check-ups, select “Health Checks”. Scroll to the type of Health Check using the Arrow button  and press the OK button . The entries for each will appear on the display.

Health Checks		
Ketones	trace	Sep 3- 02▲
B.P.	120/80	Sep 3- 02
Weight	150.0	Sep 3- 02
Height	67.0	Sep 3- 02
Dr. Visit	done	Sep 3- 02
HbA1c	5.2%	Sep 3- 02
Chol	187	Sep 3- 02▼

Health Checks	
HbA1c:	
Sep 3- 02	5.2%
Jul 14- 02	5.9%

SOURCES FOR HELP



Information

Contact
LifeScan
Customer Service
or
visit the Website at:
www.OneTouchDiabetes.com

This choice prompts you to contact LifeScan Customer Service or visit our Web site when you need help with testing. Call **LifeScan Customer Service at 1 800 227-8862. Contact us 7 days a week, 8 a.m. – 10 p.m. Eastern Time.**

Data Downloading

Transferring test results to your personal computer for home viewing requires OneTouch® Diabetes Management Software from LifeScan and a OneTouch® Interface Cable. OneTouch® Diabetes Management Software and a OneTouch® Interface Cable are not included with your OneTouch® UltraSmart® System. For order information and to learn more about OneTouch® Diabetes Management Software, visit www.OneTouchDiabetesSoftware.com.

STEP 1 Install OneTouch® Diabetes Management Software.

Install OneTouch® Diabetes Management Software on your computer following the instructions provided in the OneTouch® Diabetes Management Software User Manual.


STEP 2 Connect the OneTouch® Interface Cable.


Connect the OneTouch® Interface Cable to a serial port on the back of your computer. With the meter turned on, connect the OneTouch® Interface Cable to the data port located on the upper left side of the meter. Be sure the cable plug is inserted all the way.



Note: While in the communication mode, you will be unable to perform a blood glucose test.

STEP 3 Transfer Data.

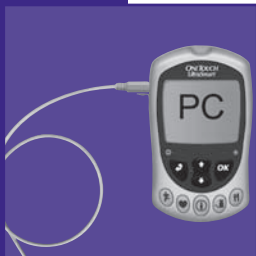

Follow the instructions in the OneTouch® Diabetes Management Software User Manual to initiate the data transfer command. “PC” will appear on the display once the first command is received, indicating that the meter is in the communication mode. You are now ready to transfer all test results—with date and time—stored in the meter memory to your computer. If the command is not received within one minute, the meter will turn itself off. You may also press and hold the Back button  to turn off the meter.

For order information and to learn more about OneTouch® Diabetes Management Software, visit www.OneTouchDiabetesSoftware.com or call **LifeScan Data Management Line at 1 800 382-7226.**

SPECIFICATIONS

Result Range:	20 to 600 mg/dL
Calibration:	Plasma-equivalent
Sample:	Fresh capillary whole blood
Sample Size:	Minimum 1 microliter (μL)
Test Time:	5 seconds
Assay Method:	Glucose oxidase biosensor
Power Source:	Two AAA alkaline batteries
Battery Life:	About six months at three tests per day (one test using backlight)
Glucose Units:	mg/dL
Memory:	Over 3,000 records
Automatic Shutoff:	Three minutes after inserting a test strip if sample has not been applied; one minute after all other user actions
Size:	Approx. 3.8 in. (L) x 2.3 in. (W) x 0.9 in. (H)
Weight:	2.8 ounces (with batteries)
Operating Ranges:	Temperature 43–111°F/6–44°C Relative humidity 10–90% Hematocrit 30–55%

PRODUCT SAFETY INFORMATION

The OneTouch® UltraSmart® System complies with the following standard:

UL (Underwriters Laboratories Standard) UL 60601-1

- Internally powered equipment. Battery operated with two 1.5 volt AAA alkaline batteries.
- This unit is not suitable for use in the presence of flammable anesthetic mixture with air, oxygen or nitrous oxide.
- Continuous mode of operation.
- IPX4 degree of protection against ingress of water. This is a level of water resistance (protection against splashing water).
- Type BF applied part degree of protection against electrical shock. This is a level of electrical safety protection for the user suitable for this medical device.

EXPLANATION OF SYMBOLS ON THE METER



See Accompanying Documentation. Read and understand your Owner's Booklet and test strip package insert before using this system.



This indicates type BF applied part degree of protection against electrical shock. This is a level of electrical safety protection for the patient or user suitable for this medical device.



This indicates electrical safety certification by the Canadian Standards Association (CSA) in accordance with UL 60601-1, CAN/CSA 22.2 No. 601.1-94 and CAN/CSA 22.2 No. 601.1.2-94.

GUARANTEE

The makers guarantee that the OneTouch® UltraSmart® Meter shall be free of defects in material and workmanship for a period of three years. This guarantee is valid from the date of purchase. The guarantee extends only to the original purchaser and is not transferable.

INDEX

Advanced Features	72–76
Alternate Site Testing	15, 21–23, 27–30, 32
Arrow Button	3
Averages, All Results	109, 113
Averages, By Exercise	109, 114
Averages, By Time of Day	109, 114
Averages, Food	109, 121
Averages, Insulin Daily	117, 119
Back Button	3
Backlight	3
Backlight Button	3
Basic Set-up Mode	58–61
Batteries	1, 43–45, 55, 125–126
Beep	64, 66–67
Blood Glucose Testing	19–39
Blood Sampling, Fingertip	23–26
Blood Sampling, Forearm or Palm	27–30
Carrying Case	1
Checking the System with Control Solution	12–18
Cholesterol Units	64, 67
Code Numbers	10–12, 15, 31, 46–47
Coding the Meter	10–12
Comments	36, 39, 77–81
Comments, Exercise	81
Comments, Food	79–80
Comments, Health	80–81

Comment? Press OK	36
Confirmation Window	7, 34–35
Contact Bars	7, 31
Control Solution, Before you begin	13
Control Solution, Checking the System with	12
Control Solution, OneTouch® Ultra®	1, 12–18
Control Solution, Out-of-range results	17–18
Custom Meter Set-up Sequence	62–76
Data Downloading	122–124
Data Port	2, 123
Date and Time, Entering	85
Date and Time, Setting	60–61
Date Format	64, 66
Dehydration	iii, 41
Diabetes Pill Set-up	68–69
Diabetes Pills	96
Display Contrast	6
Display Language	59
Display, Meter	2, 9
Display Symbols	6
Editing Comments	103–105
Editing Logbook Entries	105–107
Electronic Logbook	i, 77–107
Entries, Logbook	82–107
Error Messages	51–55
Exercise Comments	81
Exercise Logbook Entries	86–87

Exercise SmartButton®	4-5, 77, 82-83, 86
FastFacts®	108-122
FastFacts® Feature	108-122
FastFacts® Menu	108-109
FastFacts® SmartButton®	4-5, 82, 108
Fingertip Testing Procedure	23-26
Food Averages	109, 121
Food Comments	79-80
Food Logbook Entries	101-102
Food SmartButton®	4-5, 77, 82-83
Forearm	21-22, 27, 30, 33
Forearm or Palm Sampling,	
Important Information	21-22
Format, Date	64, 66
Format, Time	64, 66
Glucose Analysis	109, 111-115
Glucose by Meals	109-110
Glucose Graph, Interactive	111-113
Glucose Range	73, 76, 115
Graph, All Results	109, 111-112
Graph, Time of Day	109, 112-113
Guarantee	127
Health Checks	87-94, 109, 121
Health Comments	80-81
Health Logbook Entries	87-94
Health Notes	87-88
Health SmartButton®	4-5, 77, 82-83

High Glucose	iii, 38, 51
Hyperglycemia	iii, 38–39, 50–51
Hypoglycemia	iii, 21–22, 38–39, 50–51
Hypoglycemia Information (Hypo Info)	109, 120
Hypoglycemia Level	73, 76
Insulin Dosages	97
Insulin Intake	116
Insulin Pump	98–100, 118–120
Insulin Pump Set-up	72
Insulin Set-up	70–71
Intended Use	ii
Interface Cable, OneTouch®	122–124
Laboratory Results	40–41
Lancets	1, 19–20, 24, 28, 37
Lancet Disposal	37
Logbook	i, 4, 77–107, 109–110
Logbook, Electronic	i, 77–107
Logbook Entries	82–107
Logbook Entries, Editing or Deleting	103, 105–107
Logbook Entries, Exercise	83, 86–87
Logbook Entries, Food	83, 101–102
Logbook Entries, Health	83, 87–94
Logbook Entries, Medication	83, 95–100
Low Glucose	iii, 38, 51
Maintenance, Meter	42
Maintenance, OneTouch® UltraSoft® Adjustable Blood Sampler	42

Meal Schedule	72–75
Medication Logbook Entries	83, 95–100
Medication Menu	95
Medication SmartButton®	4–5, 77, 82–83
Memory	77–107
Message, Check Ketones	39, 50
Message, Snack	39, 50
Messages, Display	46–50
Messages, Error	51–55
Messages, Special	38–39
Messages, Warning	51–55
Milligrams per Deciliter	9, 17, 36, 49, 125
Off Button	3
OK Button	3
OneTouch® Diabetes Management	
Software	64, 68–70, 122–124
OneTouch® Interface Cable	122–124
OneTouch® Ultra® Control Solution	1, 12–18
OneTouch® UltraClear® Cap	1, 19, 28–30
OneTouch® Ultra® Test Strips	1, 7–8
OneTouch® UltraSmart® Meter	1–3
OneTouch® UltraSmart® System	1–8
OneTouch® UltraSoft® Adjustable	
Blood Sampler	1, 19–20, 23–30
Owner's Booklet	1
Palm	21–22, 27, 30, 33
Problems and Possible Causes	56–57

Product Safety Information	126
Quick Reference Guide	1
Quick Start Guide	1
Reduce Chance of Infection	20
Repeated Unexpected Results	iii
Schedule, Meal	72–75
Settings	66–67
Set-up, Basic Meter	58–61
Set-up, Custom Meter	62–76
Set-up Menu	62–63
Specifications	125
Syringe Totals	117–118
System Overview	i–8
Temperature Range	8, 13, 36, 53, 55, 125
Test Mode	15, 32
Test Port	2, 31
Test Procedure	31–35
Time Format	64, 66
Time Setting	60
Unexpected Results	iii
Unit of Measure, mg/dL	9, 17, 36, 49, 125
Vial, Control Solution	12–13, 16–18
Vial, Test Strip	8, 10–12, 17
Warning Messages	51–55
Warranty Registration Card	i, 1
Week, Beginning of	64, 66–67

Notes

Notes

Notes

As your partner in diabetes care, we offer valuable diabetes-related knowledge, tools and special offers online.

www.OneTouchDiabetes.com
www.OneTouchEnEspanol.com

Should you need additional assistance, we welcome you to contact us 7 days a week, 8 a.m. – 10 p.m. Eastern Time.

1 800 227-8862 (English)
1 800 381-7226 (Español)

For assistance outside of these hours, please contact your healthcare professional.

Distributed by:
LifeScan, Inc.
Milpitas, CA 95035

For:
LifeScan Europe
Division of Cilag GmbH International
6300 Zug, Switzerland

© 2006 LifeScan, Inc.
Rev. date: 02/2011 AW 06078808B

Meter Made in China



AW 06078808B