


# Inserting or Replacing the Batteries

The CareSens N Meter uses two 3.0 V lithium batteries. Before using the meter, check the battery compartment and insert batteries if empty.

When the  symbol appears on the display while the meter is in use, the batteries should be replaced as soon as possible. The test results may not be saved if the batteries run out completely.



## Step 1

Make sure the meter is turned off. Push the cover in the direction of the arrow to open the battery compartment.



## Step 2

Remove the old batteries one by one by lifting with the index finger and pulling it out with your thumb and index fingers as shown in the diagram on the right. Insert two new batteries with the + side facing up and make sure the batteries are inserted firmly.

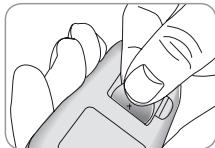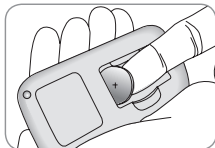

## Step 3

Place the cover on the battery compartment. Push down until you hear the tab click into place.



**Note:** Removing the meter batteries will not affect your stored result. However, you may need to reset your meter settings. See pages 14-16 .