

## CareSens N Blood Glucose Meter

### Data Port


Used to transfer data from the meter to a computer with the cable available from your authorised i-SENS representative

### ↑ Button

Selects or changes information

### S Button

Turns the meter on/off, confirms menu selections, and changes information

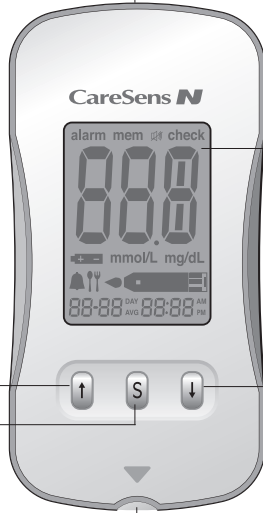

**Test Strip Port**  
Insert test strip here

### Display

Shows results, messages

### ↓ Button

Selects or changes information

**Note:** The cable for data transmission to PC can be ordered separately. Please contact your authorised i-SENS sales representative.

## CareSens N Blood Glucose Meter Display

### mem

appears when test results stored in the memory are displayed

### alarm

appears when the time alarm has been set

### Battery symbol

indicates meter battery is running low and needs to be replaced

### Post-meal test flag

appears during post-meal testing and when post-meal test results are displayed

### Alarm symbol

appears when the post-meal alarm has been set

### mmol/L

unit for measuring blood glucose

### Sound symbol

appears only when the sound is set to OFF

### check

appears when the control solution test results are saved or displayed

### Test results

test results displaying panel

### Decimal point

appears when the blood glucose measuring unit is set to mmol/L

### mg/dL

unit for measuring blood glucose

### Blood insertion symbol

indicates meter is ready for the application of a drop of blood or control solution

**Note:** The unit of measurement is fixed in mmol/L and it cannot be changed to mg/dL by the user.

