

Setting up Your System

Press and hold the **S** button for 3 seconds to enter the SET mode. After all settings are finished, press and hold the **S** button for 3 seconds to turn off the meter.

Press **↑** or **↓** to reach the correct value. Press and hold **↓** to scroll faster.

Adjusting the Date and Time

Step 1 Entering the SET Mode

Press and hold the **S** button for 3 seconds to enter the SET mode. After all the segments flash across the screen, the 'SET' character will be displayed on the screen. Press the **S** button again to progress to setting the year.



Step 2 Setting the Year

Press and release **↑** or **↓** to adjust until the correct year appears. Press and hold **↓** button to scroll through the numbers quickly. After setting the year, press the **S** button to confirm your selection and progress to setting the month.



Step 3 Setting the Month

A number indicating the month will be blinking on the left corner of the screen. Press **↑** or **↓** until the correct month appears. Press the **S** button to confirm your selection and progress to setting the date.




Step 4 Setting the Date

Press **↑** or **↓** until the screen displays the correct date. Press the **S** button to confirm the date and progress to setting the time.



Step 5 Setting the Time

The meter can be set in the AM/PM 12-hour or the 24-hour clock format. Press **↑** or **↓** to select a format. The AM • PM symbol is not displayed in the 24-hour format. After selecting the format, press the **S** button to progress to setting the hour.

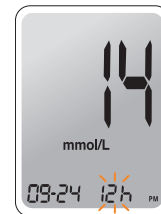

Step 6 Setting the Hour

Press **↑** or **↓** until the correct hour appears. After the hour is set, press the **S** button to progress to setting the minute.



Step 7 Setting the Minute

Press **↑** or **↓** until the correct minute appears. After setting the minute, press the **S** button to progress to setting the sound.



Setting the Sound On/OFF

Step 8

On pressing **↑** or **↓**, the screen will display On or OFF. Press the **S** button to confirm the selection.

The meter will beep in the following instances, if set to On.


- When you push the **S** button or **↑** button to turn on the meter
- When the test strip is inserted in the meter
- When the blood sample is absorbed into the test strip and the test starts
- When the test result is displayed
- When you push the **↑** button to set the post-meal (PP2) alarm
- When it is time for a preset blood glucose test



If the sound is set to OFF, none of the sound functions will work.

To confirm all settings, press and hold the **S** button and the screen will switch off.

Note:

- Only when the sound is set to OFF,  symbol appears on the display.
- At any stage, if the **S** button is pressed for 3 seconds, Time and Date setting mode will finish and the meter will be turned off. Press and hold **↓** to scroll through numbers quickly.